

How to use

How to use Clinell Universal Wipes




Wash or decontaminate your hands and put on PPE (risk assessment to ensure appropriate PPE is worn, as per local protocol).



Remove one wipe from the pack.



Working from top to bottom, clean to dirty, wipe in an 'S' shaped pattern.

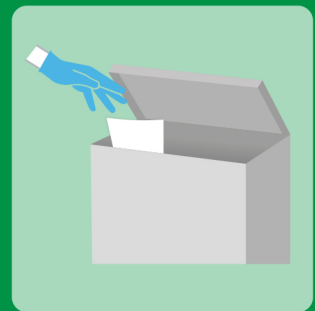

Should your wipe dry out or become soiled, replace it with a new wipe. Let the surface air dry.

How to use Clinell Universal Spray




Wash or decontaminate your hands and put on PPE (risk assessment to ensure appropriate PPE is worn, as per local protocol).



Turn nozzle and spray surface.



Working from top to bottom, clean to dirty, wipe in an 'S' shaped pattern.

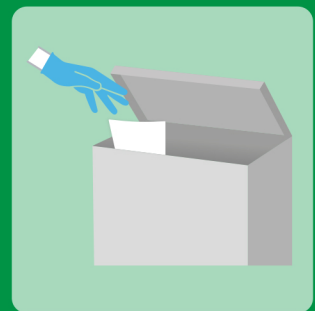

Should your wipe dry out or become soiled, replace it with a new wipe. Let the surface air dry.

The following equipment should be regularly decontaminated with Clinell Universal Wipes as per local protocol / policy.

(Examples of use - this is not an exhaustive list)

- | | | |
|--------------------------|---------------------------|------------------------|
| - Ultrasound transducers | - ECG machines | - Blood pressure cuffs |
| - Pulse oximeter probes | - Incubators | - Monitors |
| - Doppler probes | - Blood pressure machines | - Mattresses |
| - Stethoscopes | - Radiology equipment | |