

drive

 DeVilbiss[®]
HEALTHCARE

Community Bed Infusion (Drip) Pole

Instructions for use

INTRODUCTION


Thank you for purchasing the Drive DeVilbiss Healthcare Ltd. community bed infusion pole. This infusion pole is only designed for use with recommended Drive DeVilbiss Healthcare Ltd. community beds; please refer to your bed instructions for use or contact Drive DeVilbiss Healthcare Ltd. for details of compatible beds. These instructions for use cover the fitting procedure and use of the infusion pole and should be fully read and understood before the product is used.



Warning

- If there is any doubt about the assembly or suitability of the infusion pole contact the provider of the equipment or Drive DeVilbiss Healthcare Ltd.
- Before the infusion pole is fitted to the bed, a risk assessment must be performed on a patient by patient basis to assess suitability.


PART IDENTIFICATION

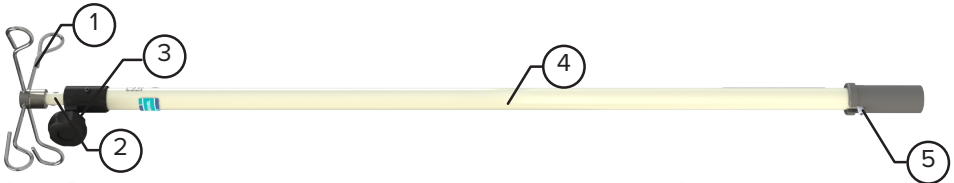

- | | |
|-------------------------------------|--------------------|
| 1. Bag hook moulding x 1 | 4. Lower tube x 1 |
| 2. Extension bar x 1 | 5. Socket plug x 1 |
| 3. Height adjustment hand wheel x 1 | |

FITTING THE INFUSION POLE

Your new infusion pole is supplied fully assembled.

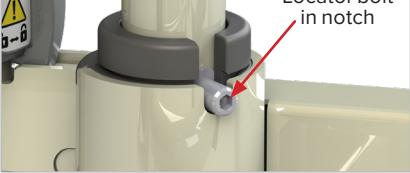1. Remove the infusion pole from any protective packaging.
2. Place the infusion pole into the desired accessory socket on the bed, positioning the locator bolt into the corresponding notch in the bed's accessory socket. The infusion pole will now be locked in place and the lower tube will not rotate.
3. To adjust the height of the bag hook, hold the extension bar with one hand to prevent it from dropping down, then loosen the height adjustment hand wheel by half a turn. Slide the extension bar to the desired height then re-tighten the height adjustment hand wheel to lock the extension bar in place. The same procedure should be used to rotate the bag hook.

Locator bolt in accessory socket notch
(image shown of BED/DP fitted to SOLITE/PRO bed)

USING THE INFUSION POLE

Each bag hook can support up to 3kg (12kg in total). Care should be taken to ensure infusion bag tubing does not become twisted around the infusion pole.

Items can be clamped on to the lower tube of the infusion pole. The total maximum load of any items clamped on to the lower tube must not exceed 10kg.

Periodically check to ensure the hand wheel remains tight.



- With the infusion pole fitted care must be taken when operating the bed. If the bed is raised or tilted the infusion pole could impact with a wall, ceiling, light fitting etc.
- The total weight of all items clamped on to the lower tube must not exceed 22kg - risk of infusion pole collapsing.
- Items must not be clamped on to the extension bar - risk of infusion pole collapsing.

MAINTENANCE

Drive DeVilbiss Healthcare Ltd. recommends that the carer performs frequent visual and operational inspections. If there are any signs of damage or the infusion pole is not performing as it should withdraw it from service until it has been repaired and is fit for use again.

Periodically check to ensure that:

- The extension tube slides freely when the height adjustment handwheel is loosened.
- The extension tube locks securely in place when the height adjustment handwheel is tightened.
- The product operates as per its intended purpose.
- The plastic mouldings are mechanically sound, no sign of cracking or damage.
- The metal fabrications are mechanically sound, no sign of cracked paint.
- No parts show signs of excessive wear.
- The product is cleaned following the guidelines in these instructions for use.

CLEANING

It is advisable to remove the infusion pole from the bed prior to cleaning.

- Wipe down with a disposable soft cloth moistened with a mild detergent and diluted in warm water (40°C).
- Rinse with cold, clean water and a clean cloth, and allow to fully dry before use.

If the product becomes soiled and the need to thoroughly disinfect is required, please refer to the bed's instructions for use for full decontamination detail. Alternatively, contact Drive DeVilbiss Healthcare Ltd. on the contact details provided overleaf.

SPECIFICATION

Product code	BED/DP
Overall height (minimum)	1300mm
Overall height (maximum)	1790mm
Overall width	245mm
Product weight	3kg
Maximum load per hook	3kg
Maximum load clamped on to lower tube	10kg

WARRANTY

Drive DeVilbiss Healthcare Ltd. warrants that this product will perform in accordance with its specification and will remain free from defects in material and workmanship when used under normal conditions for a period of 1 year (1 year full parts and labour) from the date of purchase from Drive DeVilbiss Healthcare Ltd. and its subsidiary companies. If purchased from an authorised dealer or international distributor, the product is warranted for 1 year parts only.

DRIVE DEVILBISS HEALTHCARE LTD. MAKES NO OTHER WARRANTIES, EXPRESS OR IMPLIED, AND ALL IMPLIED WARRANTIES OF MERCHANTABILITY, NON-INFRINGEMENT AND FITNESS FOR A PARTICULAR PURPOSE ARE HEREBY DISCLAIMED. IN NO EVENT WILL DRIVE DEVILBISS HEALTHCARE LTD. BE LIABLE FOR PUNITIVE, SPECIAL OR CONSEQUENTIAL DAMAGES, OR FOR AN AMOUNT IN EXCESS OF THE PURCHASE PRICE OF THE DEFECTIVE DRIVE DEVILBISS HEALTHCARE LTD. PRODUCT OR PRODUCTS.

This limited 1 year warranty gives you specific legal rights and you may also have other rights.

Drive DeVilbiss Healthcare Ltd. has a policy of continual product improvement and reserves the right to amend specifications covered in this document.

Please contact Drive DeVilbiss Healthcare Ltd. for complete warranty information.

No part of this document may be reproduced without the written approval of Drive DeVilbiss Healthcare Ltd.



CONTACT INFORMATION

Tel: +44 (0) 845 0600 333 Fax: +44 (0) 845 0600 334

Email: info@drivedevilbiss.co.uk

www.drivedevilbiss.co.uk

Drive DeVilbiss Healthcare Ltd., Sidhil Business Park, Holmfield, Halifax, West Yorkshire, HX2 9TN United Kingdom

A member of the Drive DeVilbiss Group of Companies



ORIGINAL INSTRUCTIONS

INSTRUC/BED/DP, 2023/04 - rev1