



Fitting instructions for the shell chair foot plate

Supplied parts (code: M20823400)

Foot plate and brackets

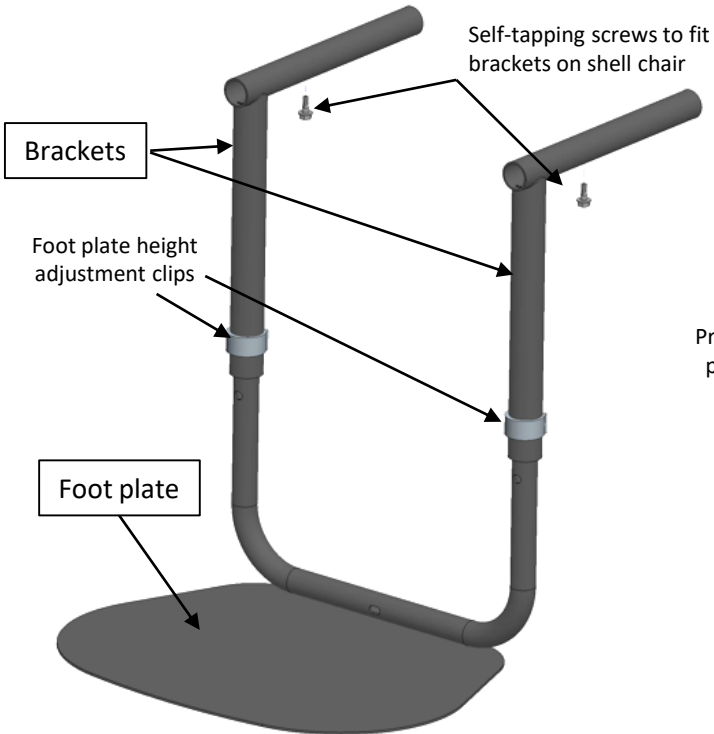

Special hinged assembly for foot plate

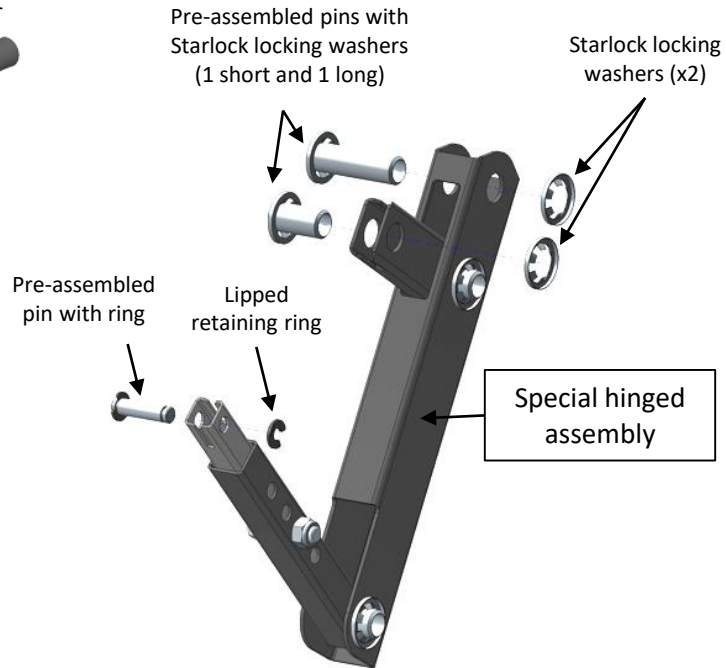

To adapt the foot plate on the shell the existing hinged assembly fit on the shell chair must be removed and changed by the above special hinged assembly.

Assembly steps


CAUTION: Wear PPE (Personal Protective Equipment) before starting operations, notably safety glasses and protective gloves



STEP 1 : Removal of the existing hinged assembly of the shell chair




1.1 Preparation:
Tilt the chair shell back to raise the leg rest

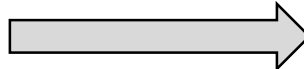


1.2 Release of the leg rest:

Remove the plastic lipped retaining ring using a small flat screwdriver

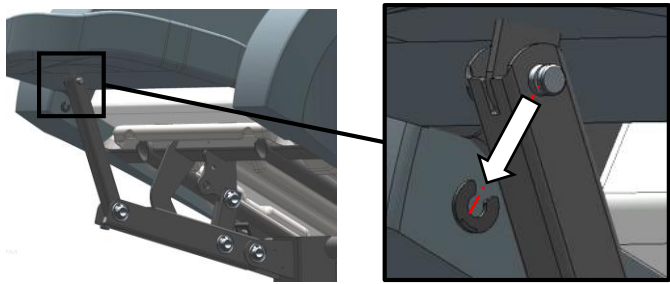

Pull of pin to take it out.

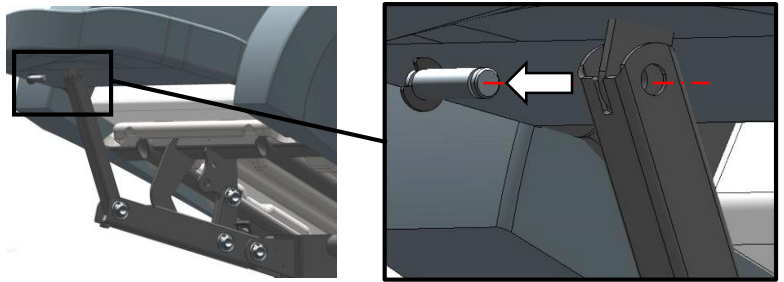


At this point, the leg rest is released from the hinged assembly

Raise the legrest for a clear view of the work area




1.3 Removal of the hinged assembly


Remove the two Starlock locking washers using a small flat screwdriver (Bend the tangs up to release the tension and allow to remove them)

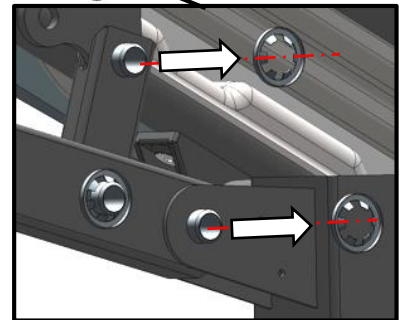

Pull off the pins to take it out

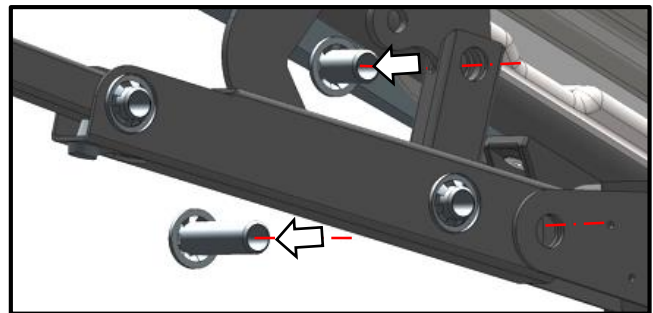


At this point, the hinged assembly can be removed out of the shell chair

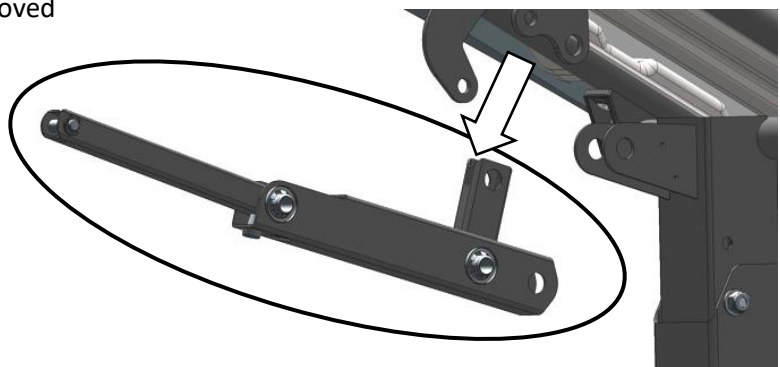


STEP 2 : Fitting of the special hinged assembly for foot plate


Put back in place the special hinged assembly supplied with the foot plate

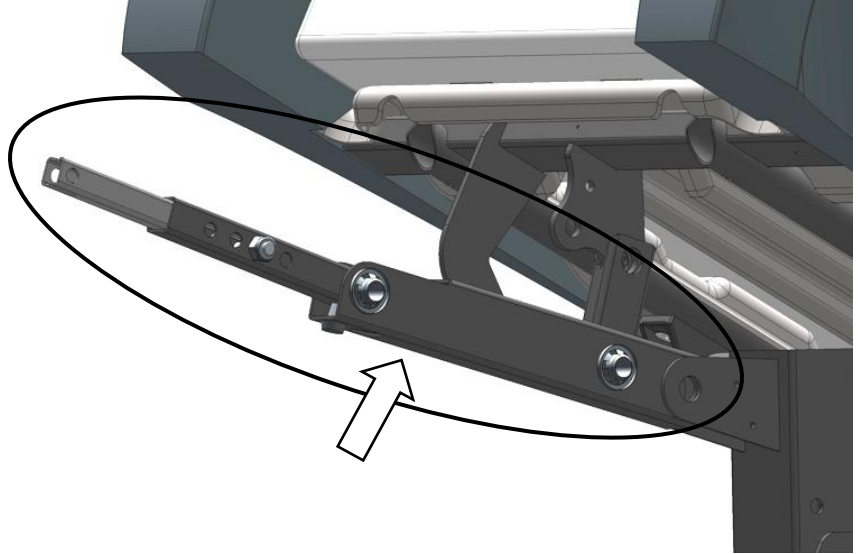

Position the pre-assembled pins supplied with the foot plate at the defined location and push on them (respect the right position of the short and long pins)

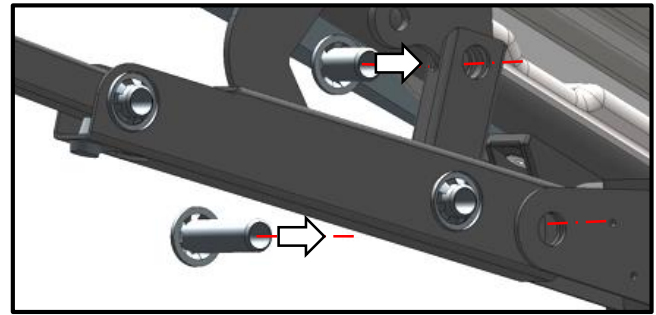


Assemble the two new Starlock locking washers supplied with the foot plate on the pins by exerting sufficient force to engage them

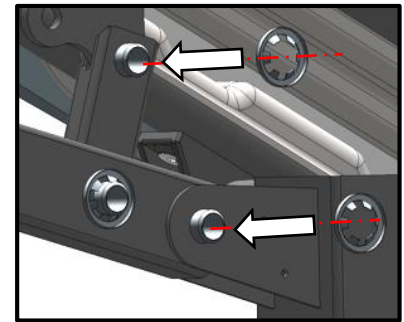



STEP 3 : Fitting of the foot plate



Insert the two brackets in the front tubes under the seat


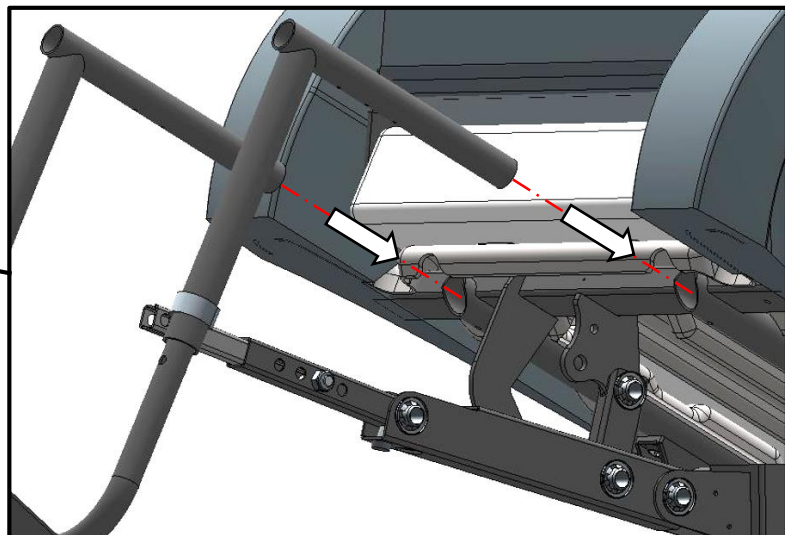


Adjust the correct depth (44cm seat depth) :
 Among the 3 fixing holes made on the support, match the middle one with the fixing hole on the tube of the chair before fitting the screws

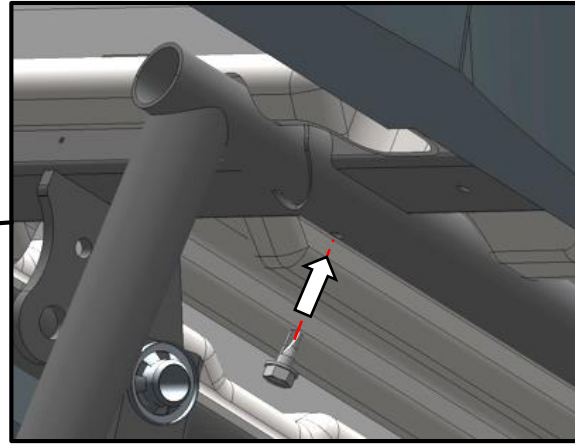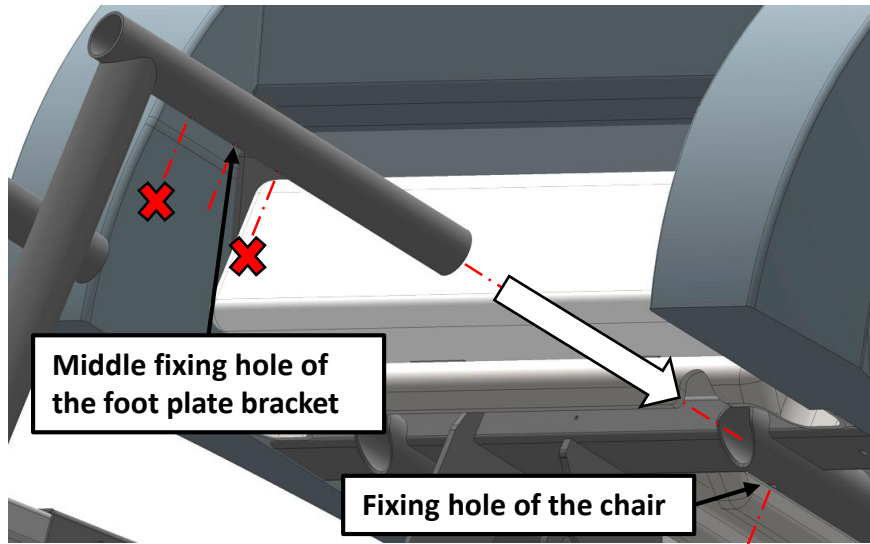



STEP 4 : Leg rest connection

Lower the leg rest and connect it to the arm of the hinged assembly

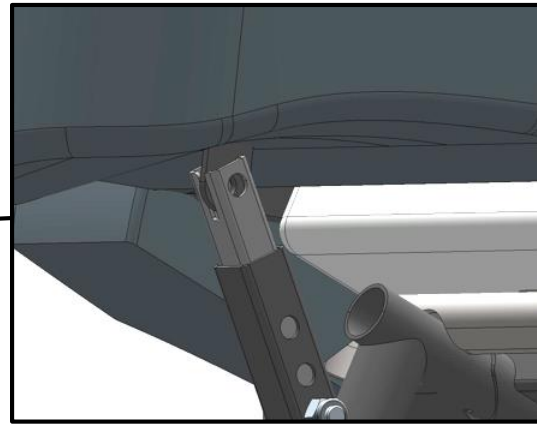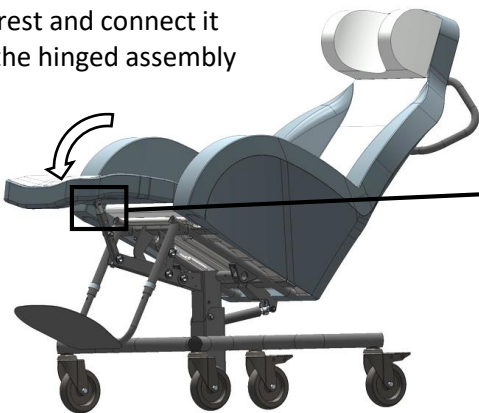

Push on the pre-assembled pin with ring


Assemble the lipped retaining ring in the axle groove

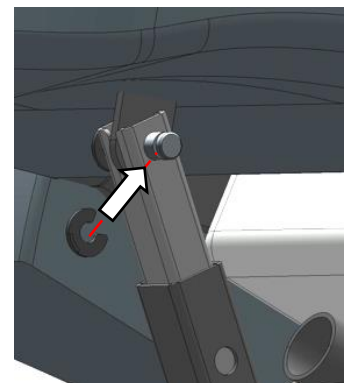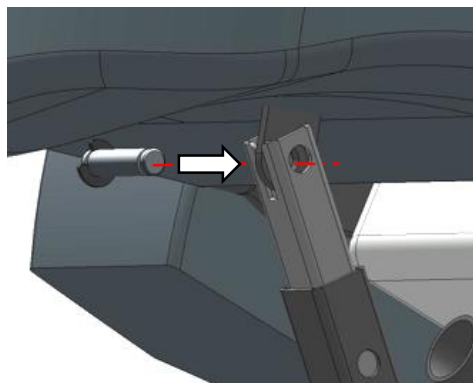

STEP 5 : Checking

Before using the wheelchair check the following points:

1/ Check that the bolt is in the 2nd hole made on the tube (44cm seat depth). Otherwise correct

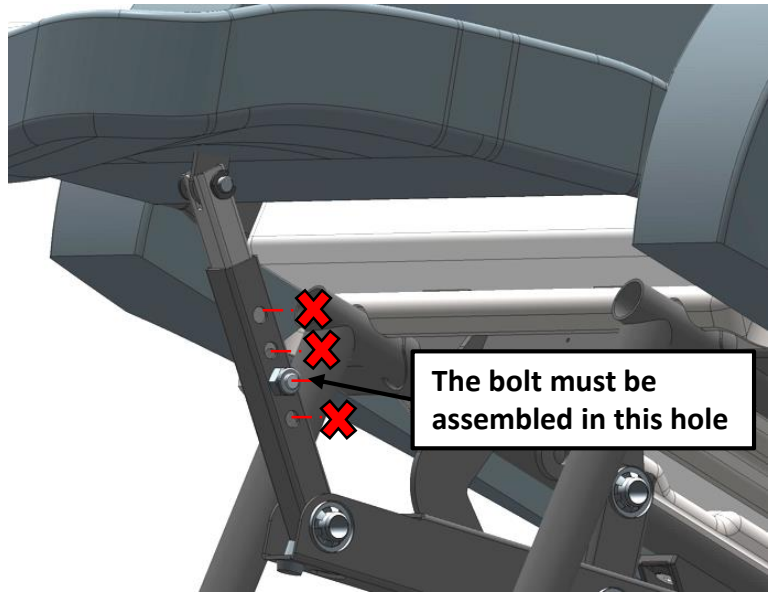

2/ Check the foot plate height adjustment:
When the seat is in the zero tilt, the foot plate must touch the ground.
Otherwise correct by removing both clips and adjusting the foot plate height before putting back the clips.

