

ENTERPRISE PT10 PT11

INSTRUCTIONS FOR USE



Contents

1. Introduction	2
2. Assembly	3
3. Operation	4
4. Cleaning and Maintenance	7
5. Warranty and Service.....	8
6. Dimensions and Data.....	9

Warnings and Cautions

WARNINGS given in this manual identify possible hazards in procedures or conditions, which if not correctly followed, could result in death, injury or other serious adverse reactions.


Cautions given in this manual identify possible hazards in procedures or conditions, which if not correctly followed, could result in equipment damage or failure.

WARNING

The carer should ensure the table is correctly positioned and adjusted to allow the patient to use it safely and comfortably.

Do not lean or sit on the table.

Do not exceed the Safe Working Load of 25kg evenly distributed.

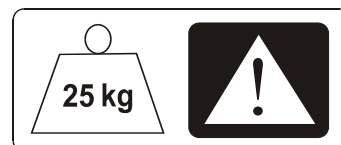

Caution

Take care when moving the table over rough or uneven surfaces. Lift the table over large ridges and wide gaps.

1. Introduction

The **Renray PT10** and **PT11** are overbed / overchair tables that provide patients with a firm, flat surface suitable for eating, writing, reading, etc. Single-column construction and adjustable height makes them suitable for use with most types of beds and chairs.

Model PT10 have a fixed one-piece top surface.


Model PT11 have a two-piece top surface with adjustable tilt.

All models have the following features:

- Wipe-clean top with raised moulded edges
- Base design allows the table to be used over either a bed or a chair
- Gas spring assisted height adjustment
- Castors for ease of mobility
- Range of colours and finishes



Model PT10 with fixed top

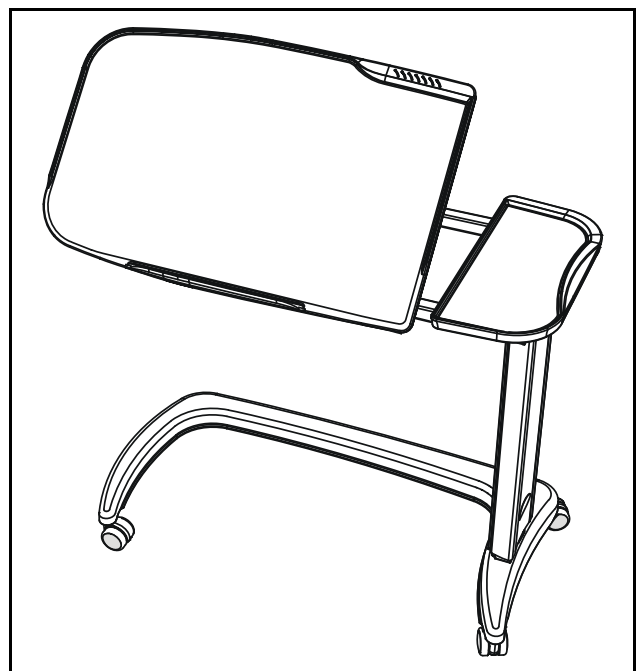


Model PT11 with tilting top

2. Assembly

General

The table may be supplied with the castors packed separately to prevent transit damage.

Fit the four castors by pushing the castor stems (A) into the sockets on the table base (B) until the castors lock into position.

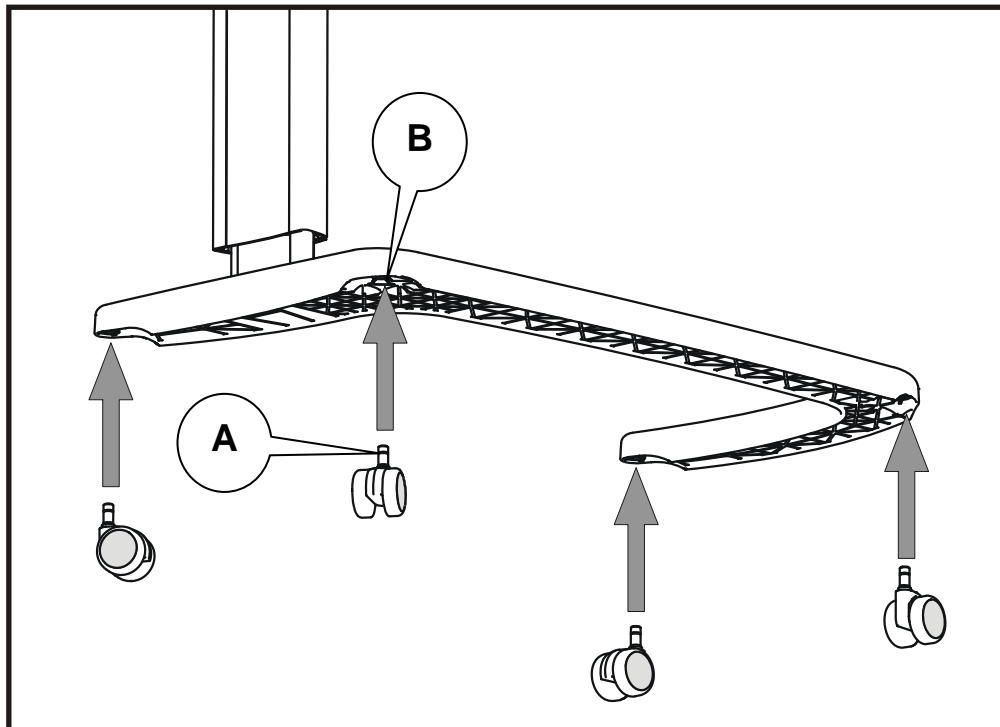

Fitting the castors

3. Operation


Height Adjustment

To lower the table height:

Lift the height adjustment lever (C) and push the table down.

To raise the table height:

Lift the height adjustment lever (C) and allow the table to rise under the action of the internal gas spring.

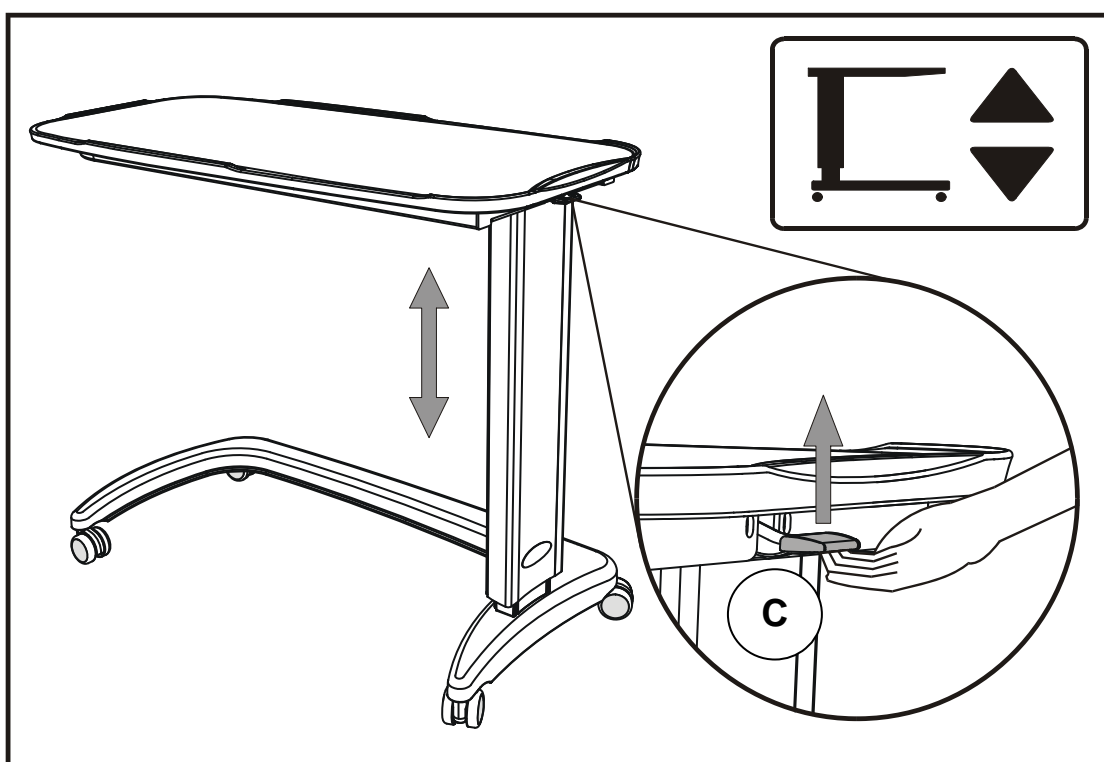

Height adjustment

Tilt Adjustment (model PT11)


The tilt angle of the table may be adjusted to one of six positions.

To increase the tilt angle:

Hold the edge of the table as shown (D). Lift the table until the ratchet (E) engages at the required position.

To reduce the tilt angle:

Hold the edge of the table as shown (D). Lift the table as far as it will go, then lower it back to the flat position.

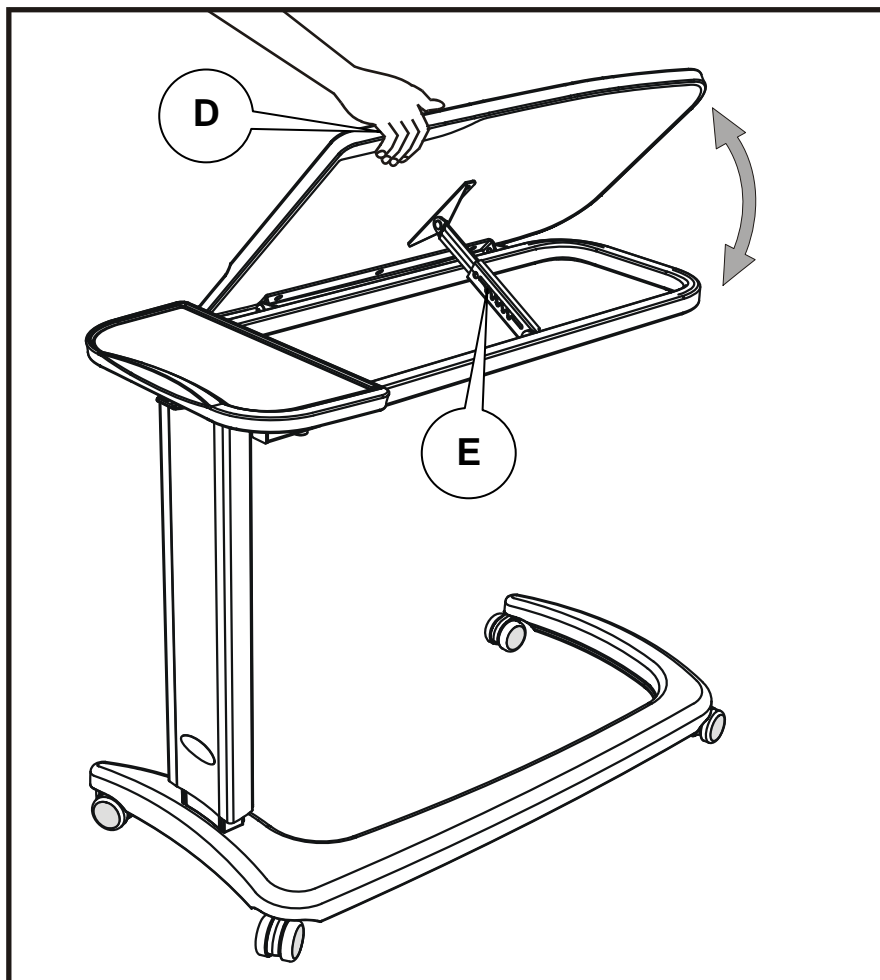

Tilt adjustment

WARNING

Take care not to trap hands or fingers between the fixed and movable parts of the table

4. Cleaning and Maintenance

Cleaning

Wearing suitable protective clothing, clean all surfaces with a disposable cloth soaked in a neutral detergent and hand hot water.

Start by cleaning the upper sections of the table and work along all horizontal surfaces. Work methodically towards the lower sections of the table and clean the castors last. Take extra care to clean areas that may trap dust or dirt.

Rinse with clean water and dry with disposable paper towels.

Disinfecting

After cleaning the table as described above, wipe all surfaces with sodium dichloroisocyanurate (NaDCC) at a concentration of 1,000 parts per million (0.1%) of available chlorine.

In the case of pooling body fluids, e.g. blood, the concentration of NaDCC should be increased to 10,000 parts per million (1%) of available chlorine.

Caution

Avoid excessive wetting of the product, particularly the table top

Do not use abrasive compounds or pads

Do not use phenol-based disinfectants

Maintenance

The following preventive maintenance procedures should be carried out at intervals of 12 months. All service or repair activities must be carried out by properly qualified and trained persons approved by Renray Healthcare.

Check that the height adjustment mechanism functions smoothly and that the table locks securely in position over the full range of table height.

Make sure all four castors are securely fixed to the base and that they swivel and rotate freely. Remove any debris that may have been picked up by the wheels.

Model PT11 only: check that the table locks securely at each tilt position.

5. Warranty and Service

Renray Healthcare's standard terms and conditions apply to all sales. A copy is available on request. These contain full details of warranty terms and do not limit the statutory rights of the consumer.

For service, maintenance and any questions regarding this, or any other Renray Healthcare product, please contact:

Renray Healthcare

Road Five

Winsford Industrial Estate

Winsford

Cheshire

CW7 3RB

Telephone 01606 593456

Fax 01606 861354

E-mail sales@renrayhealthcare.com

Internet www.renrayhealthcare.com



6. Dimensions and Data

PT10 and PT11 Overbed / Overchair Tables

Table top dimensions	90 x 42 cm max.
Height adjustment range	71 to 108 cm
Tilt angles (approx.)	0°/24°/31°/37°/42°/47° (<i>PT11 only</i>) 17 kg
Product weight (approx.)	
Safe working load	25 kg evenly distributed



Renray Healthcare
Road Five
Winsford Industrial Estate
Winsford
Cheshire
CW7 3RB

Telephone	01606 593456
Fax	01606 861354
E-mail	sales@renrayhealthcare.com
Internet	www.renrayhealthcare.com



As our policy is one of continuous development we reserve the right to modify designs and specifications without prior notice.