

Unigloves Hand Disinfection

Alcohol-based disinfectant

Biocide. Ready-to-use alcohol-based disinfectant for hygienic and surgical hand disinfection. To combat harmful organisms, dermatologically tested.

Composition:

100 g contain: 62.8 g
2-propanol

Product Code:

97-250 | 250ml
97-500 | 500ml
97-101 | 1000ml
97-105 | 5 litre

Benefits

- Dries quickly without leaving a greasy residue
- Fragrance-free variants available
- Short exposure time
- VAH-certified (hygienic hand disinfection: 30 sec., surgical hand disinfection: 3 min.)
- IHO-listed (hygienic hand disinfection: 30 sec., surgical hand disinfection: 90 sec.)
- Without QAC
- Particularly suitable for use with gloves
- Vegan
- Ethanol-free

Application Instructions

For hygienic hand disinfection, the hands are rubbed with 3 ml of the solution and kept moist for the exposure times listed below.


For surgical hand disinfection, hands and forearms are rubbed with the solution and kept moist for at least 90 seconds according to the exposure times listed below. For cooling compresses, the solution should be diluted with equal parts water.

Efficacy and test methods		Load	15 Sec	30 Sec	90 Sec
General					
Hygienic hand disinfection	EN 1500, VAH	High		3	
Surgical hand disinfection	EN 12791, VAH	High			3
Bacteria					
TbB (M. terrae)	EN 14348	High		3	
Viruses					
Enveloped viruses (e.g. HBV, HIV, HCV)	RKI/DVV and EN 14476	High	3		
Noro viruses	EN 14476	High		3	

Other Information

- Only dispose of completely empty packaging for recycling. Dispose of filling material residues under EAK 07 06 99.
- This product is intended for professional use. For further information, please refer to the safety data sheet or the product label.
- Use disinfectant with care. Always read the label and product information before use.
- Any serious incident that has occurred in relation to the device shall be reported by the user/patient to the manufacturer and the competent authority of the Member State in which the user/patient is established.
- All information is based on our current knowledge and experience and is provided for information purposes only. No warranty and/or guarantee is given for the information provided here. Information on the suitability and application of the products does not constitute a binding agreement regarding the contractual quality or specific suitability for use of the product. Contractual provisions, in particular agreed product specifications, always take precedence. We reserve the right to change or update the information on this product without prior notice.

pH value: neutral

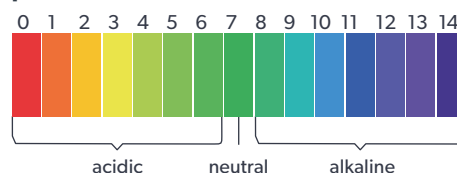

UNIGLOVES UK
Registered in England No: 4010200
VAT registration 791817787

3 Ambley Green,
Gillingham Business Park,
Gillingham, ME8 0NJ, UK
+44 (0)800 049 6602
enquiries@unigloves.co.uk
uk.unigloves.com

UNIGLOVES GERMANY
Arzt- & Klinikbedarf
Handelsgesellschaft mbH

Camp-Spich-Straße 71
D-53842 Troisdorf,
Germany
+49 (0) 2241-9323-0
unigloves@unigloves.de
de.unigloves.com



Issue No. 3.0
23/06/2026