

Unigloves disinfectant wipes Plus Soft

Fresh (neutral) and Lemon Fresh

Intended use as a medical device. Ready-to-use rapid disinfectant for the prophylactic surface disinfection of non-invasive medical devices Suitable for the medical sector (dental and medical practices, hospitals, care homes, nursing and emergency services) and the food, industrial and public sector.

Composition:

- Per 100g: 20.0g 1-propanol;
5.0g 2-propanol
- Regulation (EC) No 648/2004:
Contains fragrances

Product Code:

97-9190 Lemon Fresh
97-9290 Fresh (Neutral)
(90 Sheets/Refill)



Other notes

- Only place the packaging in the recycling bin once it has been completely emptied. Dispose of any remaining contents under EAK 07 06 99.
- This product is intended for professional use. Please refer to the safety data sheet or the product label for further information.
- Use disinfectants with care. Always read the label and product information before use.
- The user/patient must report all serious incidents relating to the product to the manufacturer and to the competent authority of the Member State in which the user/patient is based.
- All information is based on our current knowledge and experience and is provided for information purposes only. No warranty and/or guarantee is given in respect of the information provided here. Information regarding the suitability and use of the products does not constitute a binding agreement regarding the contractual quality or any specific suitability for use of the product. Contractual provisions, in particular agreed product specifications, shall always take precedence. We reserve the right to amend or update the information relating to this product without prior notice.

Benefits

- Excellent material compatibility
- Suitable for disinfecting sensitive surfaces on electronic devices, e.g. screens, touchscreens on tablets and mobile phones, and acrylic glass (PMMA)
- Very broad and rapid microbiological efficacy with a significantly reduced alcohol content – lint-free non-woven fabric

- Optimal tear-off properties Thanks to a low alcohol content of just 25%, excellent material compatibility is guaranteed even with alcohol-sensitive materials such as acrylic glass (PMMA) or imitation leather
- Virtually residue-free
- fragrance-free (when used the Fresh (Neutral) variant)

- Short contact time
- VAH-certified
- IHO-listed
- Phenol-free
- rapid action
- QAV-free
- Aldehyde-free

Efficacy and test methods		Contamination	1 min.	5 min.
Bacteria and Candida albicans	EN 13727, EN 13624	High	3	
Tuberculosis pathogens (M. terrae) and mycobacteria (M. avium)	EN 14348	High	3	
"Limited virucidal activity" (enveloped viruses such as HBV, HIV, HCV)	EN 14476	High	3	
"Limited virucidal Plus" (enveloped viruses such as HBV, HIV, HCV and noroviruses, adenoviruses and rotaviruses)	EN 14476	Low	3	
Polyoma SV 40	RKI/DVV	With contamination	3	
Pre-moistened wipes (100% PET)				
Bacteria and Candida albicans	EN 16615	High		3

Instructions for use

Disinfection of non-invasive medical devices:

- Suitable for prophylactic wipe disinfection of medical devices such as dental treatment units, hospital beds, blood pressure monitors, venous catheters, etc. To disinfect, wipe the surface of the medical device with the impregnated wipe.
- Peel the 'Stick & Lift' label off the pouch and stick it onto the dispenser. Record the opening and expiry dates in the designated field and observe them. Open the top of the pouch and place it into the dispenser. Pull out the first wipe from the non-woven roll and thread it through the star-shaped opening in the lid. Remove the remaining wipes. After removing the wipes, press the lid firmly onto the container and seal it tightly. Once all the wipes have been used, the dispenser must be cleaned and disinfected in accordance with the reprocessing instructions. Reprocessing should be carried out using a suitable, preferably alcohol-based, disinfectant wipe. All surfaces to be disinfected must be completely saturated with the disinfectant solution.
- Allow the product to act for the specified contact times. Good material compatibility with alcohol-sensitive materials. Intended for professional use only. Once the pouch has been opened for the first time, the wipes can be used for a maximum of 28 days.



UNIGLOVES UK
Registered in England No: 4010200
VAT registration 791817787

3 Ambley Green,
Gillingham Business Park,
Gillingham, ME8 0NJ, UK
+44 (0)800 049 6602
enquiries@unigloves.co.uk
uk.unigloves.com

UNIGLOVES GERMANY
Arzt- & Klinikbedarf
Handelsgesellschaft mbH

Camp-Spich-Straße 71
D-53842 Troisdorf,
Germany
+49 (0) 2241-9323-0
unigloves@unigloves.de
de.unigloves.com



Issue No. 3.0
23/06/2026