

Unigloves Disinfection of Extraction Systems Concentrate

Low-foaming concentrate for the daily cleaning and disinfection of dental suction systems and mouthwash basins. Intended for use in the (dental) medical sector.

Composition:


Regulation (EC) No 648/2004:

< 5% non-ionic surfactants,
< 5% cationic surfactants,
Fragrances

Product Code:

97-4405 | 5 Litre

97-4410 | 10 Litre

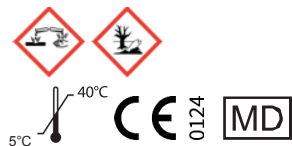

Benefits

- Works on the basis of quaternary ammonium compounds
- Extremely low-foaming formulation
- Gentle on materials
- Quick-acting
- Pleasant odour
- Free from aldehydes and phenols
- VAH-certified

Application Instructions


The concentrate must first be diluted with water. To do this, add tap water to 20 ml of concentrate to make up 1000 ml, and draw this through the suction hose (saliva ejector). The suction system should be disinfected once or twice a day, depending on the level of contamination. The mouthwash basin is disinfected by pouring in approx. 250 ml of the ready-to-use solution. Do not mix with cleaning agents or other disinfectants! Always prepare ready-to-use solutions fresh and use immediately! For professional use only.

Efficacy and test methods		Load	Concentration in %	2 min.	5 min.	60 min.
Bacteria and Candida albicans following VAH with a high load as instrument disinfection	VAH	High	2.0%		3	
			1.0%			3
Bacteria	EN 13727, EN 14561	High	3.0%	3		
Candida albicans	EN 13624, EN 14562	High	3.0%	3		
Enveloped viruses (e.g. HBV, HIV, HCV)	RKI/DVV, EN 14476	With contamination	2.0%	3		
		High	2.0%		3	

Other Information

- Only place the packaging in the recycling bin once it has been completely emptied. Dispose of any remaining contents under EAK 07 06 99.
- This product is intended for professional use. Please refer to the safety data sheet or the product label for further information.
- Use disinfectants with care. Always read the label and product information before use.
- All serious incidents occurring in connection with the product must be reported by the user/patient to the manufacturer and the competent authority of the Member State in which the user/patient is based.
- All information is based on our current knowledge and experience and is provided for information purposes only. No warranty and/or guarantee is given in respect of the information provided here. Information regarding the suitability and application of the products does not constitute a binding agreement regarding the contractual quality or any specific suitability for use of the product. Contractual provisions, in particular agreed product specifications, shall always take precedence. We reserve the right to amend or update the information relating to this product without prior notice.

pH value: 9

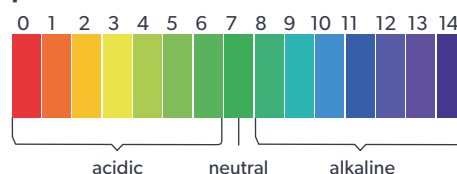

UNIGLOVES UK
Registered in England No: 4010200
VAT registration 791817787

3 Ambley Green,
Gillingham Business Park,
Gillingham, ME8 0NJ, UK
+44 (0)800 049 6602
enquiries@unigloves.co.uk
uk.unigloves.com

UNIGLOVES GERMANY
Arzt- & Klinikbedarf
Handels-gesellschaft mbH

Camp-Spich-Straße 71
D-53842 Troisdorf,
Germany
+49 (0) 2241-9323-0
unigloves@unigloves.de
de.unigloves.com

