

Unigloves Alcohol-free Surface Disinfectant

Alcohol-free surface disinfection

As a medical device: Alcohol-free, ready-to-use preparation for the prophylactic disinfection of surfaces of sensitive non-invasive medical devices. The product combines very good efficacy with outstanding material compatibility. As a biocidal product: Alcohol-free, ready-to-use wipe disinfectant for disinfecting and cleaning surfaces. Particularly suitable for use on alcohol-sensitive surfaces such as acrylic glass.

Composition:

100g contains:

- 0.24 g benzyl-C12-16-alkyl-dimethylammonium chloride
- 0.24g N,N-didecyl-N,N-dimethylammonium chloride
- 0.24g C12-14 Alkyl[(ethylphenyl)methyl]dimethylammonium chloride

Product Code:

97-6750 | 1 Litre
97-6005 | 5 Litre



Benefits

- Exceptional material compatibility, e.g., with tablets, acrylic glass or sensitive artificial leather
- Ready-to-use
- Broad effectiveness within a short time
- VAH-certified
- Aldehyde-free
- Phenol-free

Application Instructions

Intended for use in the medical sector ((dental) medical practices, clinics, retirement homes, care and rescue services) as well as in the food sector, industry and public facilities.

Efficacy and test methods		Load	Concentration in %	1 min.	2 min.	30 min.
Disinfection of small surfaces in the areas of food, industry, household and public facilities						
Bacteria and Candida albicans	EN 1276, EN 1650	High	Undiluted	↻		
Prophylactic disinfection of non-invasive medical devices and surfaces in the medical field						
Bacteria and Candida albicans	EN 13727, EN 13624	High	Undiluted	↻		
Bacteria and Candida albicans	VAH with mechanical action, EN 16615	High	Undiluted		↻	
"Limited virucidal" (enveloped viruses such as HBV, HIV, HCV)	RKI/DVV, EN 14476	High	Undiluted	↻		
Rotaviruses	EN 14476	Low	Undiluted	↻		
Noroviruses	EN 14476	Low	Undiluted			↻
Polyoma-SV 40 viruses	RKI/DVV	With contamination	Undiluted			↻


Other notes

- Only put the packaging into the recycling system if it is completely empty. Dispose of residual product in accordance with EAK 07 06 99.
- This product is intended for professional use. For further information, please refer to the safety data sheet or the product label.
- Use disinfectant with care. Always read the label and product information before use.
- Any serious incident that has occurred in relation to the device shall be reported by the user/patient to the manufacturer and the competent authority of the Member State in which the user/patient is established.
- All information is based on our current knowledge and experience and is provided for information purposes only. No warranty and/or guarantee is given for the information provided here. Information on the suitability and application of the products does not constitute a binding agreement regarding the contractual quality or specific suitability for use of the product. Contractual provisions, in particular agreed product specifications, always take precedence. We reserve the right to change or update the information on this product without prior notice.

Application Instructions

- Disinfection of medical devices: Preparation for prophylactic wipe disinfection of medical devices such as dental treatment units, hospital beds, blood pressure monitors, etc. For wipe disinfection, soak a cloth tested in accordance with EN 16615 intensively with the disinfectant solution and wipe the surfaces. Alternatively, apply the solution directly to the surface and then wipe with the appropriate cloth.
- Disinfection of surfaces: For wipe disinfection and cleaning of small surfaces, moisten a suitable cloth very well with the disinfectant solution and wipe the surfaces, or apply the solution directly to the surface and then wipe.
- All surfaces to be disinfected must be completely wetted with the disinfectant solution. Allow the preparation to take effect according to the specified contact times. Also suitable for impregnating compatible fleece cloth rolls that have been tested with the wipe disinfectant. In this case, the required impregnation quantity of the respective fleece material must be observed. Suitable for use on alcohol-compatible materials (e.g. acrylic glass). The material compatibility of new or unknown materials should be tested on a nonvisible area. To avoid streaking, the treated surfaces should be wiped with a damp (water) disposable cloth at regular intervals. The product is intended for professional users only.

pH value: 7.2

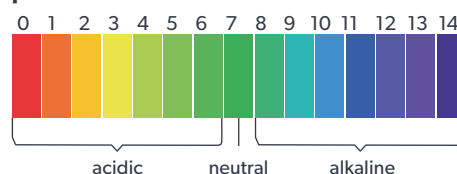

UNIGLOVES UK
Registered in England No: 4010200
VAT registration 791817787

3 Ambley Green,
Gillingham Business Park,
Gillingham, ME8 0NJ, UK
+44 (0)800 049 6602
enquiries@unigloves.co.uk
uk.unigloves.com

UNIGLOVES GERMANY
Arzt- & Klinikbedarf
Handelsgesellschaft mbH

Camp-Spich-Straße 71
D-53842 Troisdorf,
Germany
+49 (0) 2241-9323-0
unigloves@unigloves.de
de.unigloves.com



Issue No. 3.0
23/06/2026