

Unigloves Cold Spray CLASSIC

Preparation for testing the vitality of teeth

Cold Spray CLASSIC is meant for quick freezing of foam pellets which are then used for vitality testing of teeth in dentistry as well as for cooling impression materials such as hydrocolloid etc.

Product codes | Available fragrances (200 ml spray can):

9500 | Classic
9700 | Orange
9800 | Mint



Benefits

- Effective tooth vitality testing: rapid freezing of foam pellets for precise results
- Ideal for cooling impression materials such as hydrocolloid
- Easy handling: easy application from a distance of 2-3 cm
- Fast application: Icing takes place in 1-2 seconds

Other notes

- Risk of icing: Do not use directly in the mouth.
- Storage and protection: Protect from temperatures above 50°C and direct sunlight.
- Safe handling: Do not puncture or burn the container, even after use.
- Area of application: Only use in well-ventilated rooms.
- Any serious incident that has occurred in relation to the device shall be reported by the user/patient to the manufacturer and the competent authority of the Member State in which the user/patient is established.
- All information is based on our current knowledge and experience and is provided for information purposes only. No warranty and/or guarantee is given for the information provided here. Information on the suitability and application of the products does not constitute a binding agreement regarding the contractual quality or specific suitability of the product for use. Contractual provisions, in particular agreed product specifications, always take precedence. We reserve the right to change or update the information on this product without prior notice.

Application Instruction

Spray the object to be cooled from a distance of approx. 2 - 3 cm for 1 - 2 seconds. Do not use directly in the mouth, danger of freezing! Use only outside or in well-ventilated rooms. Only for professional users!



UNIGLOVES UK
Registered in England No: 4010200
VAT registration 791817787

3 Ambley Green,
Gillingham Business Park,
Gillingham, ME8 0NJ, UK
+44 (0)800 049 6602
enquiries@unigloves.co.uk
uk.unigloves.com

UNIGLOVES GERMANY
Arzt- & Klinikbedarf
Handelsgesellschaft mbH

Camp-Spich-Straße 71
D-53842 Troisdorf,
Germany
+49 (0) 2241-9323-0
unigloves@unigloves.de
de.unigloves.com

