

Unigloves Occlusion Spray

Micropowderised marking spray, light green

Occlusion Spray: Marking spray for visualizing contact points and surfaces during the fabrication of dentures

Product codes:

9601 | 75 ml Spray can



Benefits

- Precise spray film: Uniform and clear marking of even the smallest contact points
- Easy removal: Can be removed with water without leaving any residue
- Individually adaptable: Adjustable pigment layer thickness for every requirement
- Reliable and proven: No blockages, easy handling, tried and tested over many years

Other notes

- Storage: Do not store above 50°C and protect from sunlight.
- Safety measures: Do not puncture or burn pressurized containers, even after use.
- Application: Only use outdoors or in well-ventilated rooms.
- Incident reporting: Serious incidents must be reported to the manufacturer and competent authorities.
- Professional use: Only suitable for trained users in the dental field.
- All information is based on our current knowledge and experience and is provided for information purposes only. No warranty and/or guarantee is given for the information provided here. Information on the suitability and application of the products does not constitute a binding agreement regarding the contractual quality or specific suitability for use of the product. Contractual provisions, in particular agreed product specifications, always take precedence. We reserve the right to change or update the information on this product without prior notice.

Application Instruction

Instructions for use:

- Shake can well before use.
- Spray a thin layer onto the area to be examined from a distance of 2-5 cm.
- The pigment layer can be easily removed with water without leaving any residue.

Important notes:

- For professional users only.
- Do not use directly on the patient.



UNIGLOVES UK
Registered in England No: 4010200
VAT registration 791817787

3 Ambley Green,
Gillingham Business Park,
Gillingham, ME8 0NJ, UK
+44 (0)800 049 6602
enquiries@unigloves.co.uk
uk.unigloves.com

UNIGLOVES GERMANY
Arzt- & Klinikbedarf
Handels-gesellschaft mbH

Camp-Spich-Straße 71
D-53842 Troisdorf,
Germany
+49 (0) 2241-9323-0
unigloves@unigloves.de
de.unigloves.com

