

Unigloves Surface disinfection PLUS

Rapid disinfection alcoholic

As a medical device: Ready-to-use rapid disinfectant for prophylactic surface disinfection of (invasive and) non-invasive medical devices. As a biocidal product: Ready-to-use rapid disinfectant for disinfecting and cleaning smaller surfaces. Suitable for the medical sector ((dental) medical practices, clinics, retirement homes, care and rescue services) and the food, industrial and public sectors.

Composition:

100g contains:

- 20.0 g 1-propanol
- 5.0 g 2-propanol

Product Code:

97-001 Lemon Fresh | 1 Litre
 97-005 Lemon Fresh | 5 Litre
 97-010 Lemon Fresh | 10 Litre
 97-8001 Fresh | 1 Litre
 97-8005 Fresh | 5 Litre
 97-8010 Fresh | 10 Litre
 97-5001 Green Apple | 1 Litre
 97-5005 Green Apple | 5 Litre
 97-1010 Green Apple | 10 Litre



Benefits

- Excellent material compatibility
- Suitable for disinfecting sensitive surfaces of electronic devices, e.g., screens, touch surfaces of tablets, mobile phones, acrylic glass
- Very broad and rapid microbiological effectiveness with greatly reduced alcohol content
- Aldehyde-free
- Almost residue-free
- Short contact time
- VAH-certified
- Phenol-free
- The very low alcohol content of only 25% results in excellent material compatibility, even with otherwise alcohol-sensitive materials such as acrylic glass or artificial leather
- Fast effectiveness
- No QAV
- IHO- listed

Efficacy and test methods		Load	30 sec.	1 min.	5 min.
Bactericidal	EN 13727, EN 13624, EN 16615, VAH	High		3	
Candida albicans	EN 13727, EN 13624, EN 16615, VAH	High		3	
TbB (tuberculosis) pathogen (M. terrae)	EN 14348	High		3	
Mycobacteria (M. avium)	EN 14348	High		3	
"Limited virucidal" (enveloped viruses such as HBV, HIV, HCV)	EN 14476	High		3	
"Limited virucidal PLUS" (enveloped viruses such as e.g. HBV, HIV, HVC and Noro, Adeno and Rota viruses)	EN 14476	High		3	
Polyoma SV 40	RKI/DVV	With contamination		3	


Other notes

- Only dispose of completely empty packaging for recycling. Dispose of filling material residues under EAK 07 06 99.
- This product is intended for professional use. For further information, please refer to the safety data sheet or the product label.
- Use disinfectant with care. Before use always read the label and product information before use.
- All serious incidents related to the device shall be reported by the user/patient to the manufacturer and to the competent authority of the State in which the user/patient is established.
- All information is based on our current knowledge and experience and is provided for information purposes only. No warranty and/or guarantee is given for the information provided here. Information on the suitability and application of the products does not constitute a binding agreement regarding the contractual quality or specific suitability for use of the product. Contractual provisions, in particular agreed product specifications, always take precedence. We reserve the right to change or update the information on this product without prior notice.

Application Instructions

- Disinfection of non-invasive medical devices: Suitable for prophylactic wipe disinfection of medical devices such as dental treatment units, hospital beds, blood pressure monitors, tourniquets, etc. For wipe disinfection, soak a cloth tested in accordance with EN 16615 intensively with the disinfectant solution and wipe the surfaces. Alternatively, apply the solution directly to the surface and then wipe with the appropriate cloth.
- Disinfection of surfaces: For wipe disinfection and cleaning of small surfaces, wet a suitable cloth very well with the disinfectant solution and wipe the surfaces, or apply the solution directly to the surface and then wipe.

pH value: < 10

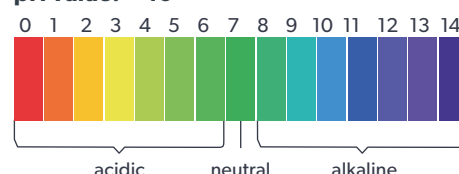

UNIGLOVES UK
 Registered in England No: 4010200
 VAT registration 791817787

3 Ambley Green,
 Gillingham Business Park,
 Gillingham, ME8 0NJ, UK
 +44 (0)800 049 6602
 enquiries@unigloves.co.uk
 uk.unigloves.com

UNIGLOVES GERMANY
 Arzt- & Klinikbedarf
 Handelsgesellschaft mbH

Camp-Spich-Straße 71
 D-53842 Troisdorf,
 Germany
 +49 (0) 2241-9323-0
 unigloves@unigloves.de
 de.unigloves.com



Issue No. 3.0
 23/06/2026