

Unigloves Impression Disinfection PLUS

Impression disinfectant

Ready-to-use, aldehyde-free impression disinfectant for dental impression materials.

Composition:

100g contains:

- 0.24 g benzyl C12–16-alkyl-dimethylammonium chloride
- 0.24 g N,N-didecyl-N,N-dimethylammonium chloride
- 0.24 g C12–14-alkyl[(ethylphenyl)methyl]dimethylammonium chloride
- 0.35 g chlorhexidine gluconate

Product Code:

97-905 | 5 Litre

97-910 | 10 Litre



Benefits

- Pleasant smell
- Excellent material compatibility
- Aldehyde- and phenol-free
- VAH-certified
- IHO-listed
- Not alcohol-based

Application Instructions

First rinse the impression in question with water, then place it in the ready-to-use disinfectant solution and remove it again after the recommended exposure time. The impression must be completely covered by the solution. Alginates and hydrocolloids should only be immersed briefly and then left to dry. Then carefully rinse the impression with a gentle stream of water. Do not mix the solution with cleaning agents or other disinfectants. The working solution must be replaced at least once per working day if visibly soiled and in accordance with RKI recommendations. It may be disposed of via the waste water system. For professional use only.

Efficacy and test methods		Exposure	5 min.	10 min.	15 min.	30 min.
Bacteria (including MRSA)	EN 13727, EN 14561, VAH	High	☞			
Candida albicans	EN 14476, EN 17111	High	☞			
'Limited virucidal activity' (enveloped viruses such as HBV, HIV, HCV)	EN 13624, EN 14562, VAH	Low	☞			
TbB (M. terrae in suspension test)	EN 14476, EN 17111	Low				☞

Other notes

- Only place the packaging in the recycling bin once it has been completely emptied. Dispose of any residual contents under EAK 07 06 99.
- This product is intended for professional use. Please refer to the safety data sheet or the product label for further information.
- Use disinfectants with care. Always read the label and product information before use.
- All serious incidents occurring in connection with the product must be reported by the user/patient to the manufacturer and the competent authority of the Member State in which the user/patient is based.
- All information is based on our current knowledge and experience and is provided for information purposes only. No warranty and/or guarantee is given in respect of the information provided here. Information regarding the suitability and use of the products does not constitute a binding agreement regarding the contractual quality or specific suitability for use of the product. Contractual provisions, in particular agreed product specifications, shall always take precedence. We reserve the right to amend or update the information relating to this product without prior notice.




UNIGLOVES UK
 Registered in England No: 4010200
 VAT registration 791817787

3 Ambley Green,
 Gillingham Business Park,
 Gillingham, ME8 0NJ, UK
 +44 (0)800 049 6602
 enquiries@unigloves.co.uk
 uk.unigloves.com

UNIGLOVES GERMANY
 Arzt- & Klinikbedarf
 Handelsgesellschaft mbH

Camp-Spich-Straße 71
 D-53842 Troisdorf,
 Germany
 +49 (0) 2241-9323-0
 unigloves@unigloves.de
 de.unigloves.com

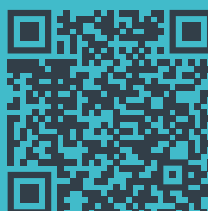

Issue No. 3.0
 23/06/2026