

Unigloves Surface Disinfectant AF Concentrate

Concentrate

As a medical device: Surface disinfectant for the prophylactic disinfection of medical devices. Not suitable for final (terminal) disinfection of invasive medical devices. As a biocidal product: Aldehyde-free concentrate for the disinfection and cleaning of surfaces. Suitable for wiping down large surfaces such as floors and walls. Also suitable for the disinfection of medical devices such as dental treatment units, surgical shoes, rubber aprons, hospital beds, blood pressure monitors, venous catheters, bedside tables, beds, walking aids, bathtubs, toilet seats, etc.

Composition:


100g contains:

- 10.8g didecyldimethylammonium chloride
- 6.0 g N-(3-aminopropyl)-N-dodecyl-propan-1,3-diamine

Labelling in accordance with Regulation (EC) No 648/2004: Contains < 5% non-ionic surfactants, 5-15% cationic surfactants, fragrances

Product Code:

97-501 | 1 Litre Dispensing Bottle
 97-505 | 5 Litre Dispensing Bottle

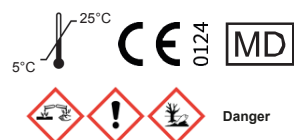

Benefits

- VAH-listed
- Excellent cleaning performance (no need for additional cleaning additives)
- Leaves no sticky residue
- Extremely broad spectrum of activity at low application concentrations
- Also suitable for dosing systems
- Free from aldehydes and phenols

Application Instructions

For the disinfection of medical devices and surfaces in healthcare settings: For wipe disinfection, thoroughly saturate a cloth tested in accordance with EN 16615 with the disinfectant solution and wipe the surfaces, or apply the solution directly to the surface and then wipe with the appropriate cloth.

Performance and test methods	Contamination	Concentration in %	5 min.	15 min.	30 min.	60 min.
Disinfection of medical devices						
Bacteria and Candida albicans	EN 13727, EN 13624, EN 16615, VAH	High	0.5			
Tuberculosis pathogen (M. terrae)	EN 14348	Low	2.0			
		High	0.5			
		High	1.0			
"limited virucidal activity" (enveloped viruses)	EN 14476, RKI/DVV	High	1.0			
		High	0.5			
"Limited virucidal PLUS"	EN 14476	Low	2.0			
Rotaviruses	EN 14476	Low	0.5			
		Low	2.0			
Adenoviruses	EN 14476	Low	1.0			
Polyoma SV 40 viruses	EN 14476	High	1.5			
Noroviruses	EN 14476	High	1.0			

Other notes


- Only place packaging in the recycling bin once it has been completely emptied. Dispose of any remaining contents under EAK 07 06 01.
- This product is intended for professional use. Please refer to the safety data sheet or the product label for further information.
- Use disinfectants with care. Always read the label and product information before use.
- All serious incidents occurring in connection with the product must be reported by the user/patient to the manufacturer and the competent authority of the Member State in which the user/patient is based.
- All information is based on our current knowledge and experience and is provided for information purposes only. No warranty and/or guarantee is given in respect of the information provided here. Information regarding the suitability and use of the products does not constitute a binding agreement regarding the contractual quality or specific suitability for use of the product. Contractual provisions, in particular agreed product specifications, shall always take precedence. We reserve the right to amend or update the information on this product without prior notice.

Application Instructions

For the disinfection of surfaces: For wipe disinfection, thoroughly saturate a suitable cloth or mop with the disinfectant solution and wipe the surfaces, or apply the solution directly to the surface and then wipe. Disinfected surfaces that come into contact with food must be rinsed with drinking water.

The concentrated disinfectant must first be diluted with tap water before use. Do not mix with cleaning products or other disinfectants! Before using on large areas of sensitive materials such as imitation leather, first test for material compatibility on a small, concealed area. All surfaces to be disinfected must be completely damp and thoroughly wetted. To prevent streaking, the treated surfaces should be wiped down with a damp cloth (water) at regular intervals. Use only as a wipe-on disinfectant! For professional use only. Unused, ready-to-use solution must be replaced depending on the degree of soiling, but at least once per working day. The used solution may be disposed of via the mains drainage system. Suitable for the medical sector (dental and medical practices, clinics, care homes, care and emergency services) and the food, industrial, domestic and public sector sectors.

pH value: > 11

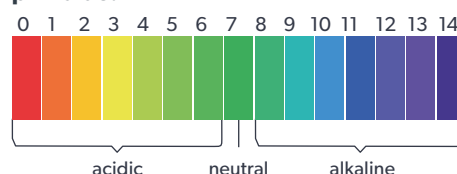

UNIGLOVES UK
 Registered in England No: 4010200
 VAT registration 791817787

3 Ambley Green,
 Gillingham Business Park,
 Gillingham, ME8 0NJ, UK
 +44 (0)800 049 6602
 enquiries@unigloves.co.uk
 uk.unigloves.com

UNIGLOVES GERMANY
 Arzt- & Klinikbedarf
 Handelsgesellschaft mbH

Camp-Spich-Straße 71
 D-53842 Troisdorf,
 Germany
 +49 (0) 2241-9323-0
 unigloves@unigloves.de
 de.unigloves.com

