

Unigloves Tray Cleaner Concentrate

Concentrate for the gentle removal of zinc oxide-eugenol pastes, carboxylate and phosphate cements, as well as alginate residues from metal and plastic surfaces. The concentrate is also suitable for removing thermoplastic impression materials and waxes, and can be used in an ultrasonic bath at room temperature.

Composition:

648/2004/EC: Between 5% and 15% anionic surfactants, fragrances.

Product Code:

97-705 | 5 Litre



Benefits

- Quickly and effectively dissolves residues
- Concentrate for use at 5–10%
- Ingredients are biodegradable
- Suitable for stainless steel items
- Solution can also be used in an ultrasonic bath

Application Instructions


Tray Cleaner is used as a 5–10% solution in water. The soiled items are best placed in the ready-to-use solution and left to soak for at least 45 minutes, depending on the degree of soiling. This specialised cleaner is suitable for non-corrosive items. Chrome-nickel and aluminium parts should not be cleaned together. Risk of corrosion!

When pre-cleaning invasive medical devices, these must be hygienically reprocessed in accordance with the manufacturer's instructions before reuse. For professional use only!

Other notes

- Only dispose of the packaging in the recycling bin once it has been completely emptied. Dispose of any remaining product under EAK 07 06 99.
- This product is intended for professional use. Please refer to the safety data sheet or the product label for further information.
- All serious incidents occurring in connection with the product must be reported by the user/patient to the manufacturer and the competent authority of the Member State in which the user/patient is based.
- All information is based on our current knowledge and experience and is provided for information purposes only. No warranty and/or guarantee is given in respect of the information provided here. Information regarding the suitability and use of the products does not constitute a binding agreement regarding the contractual quality or specific suitability for use of the product. Contractual provisions, in particular agreed product specifications, shall always take precedence. We reserve the right to amend the information relating to this product without prior notice or to update it at.

pH value: 11.3

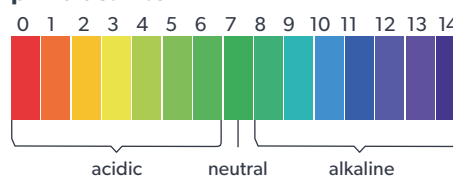

UNIGLOVES UK
 Registered in England No: 4010200
 VAT registration 791817787

3 Ambley Green,
 Gillingham Business Park,
 Gillingham, ME8 0NJ, UK
 +44 (0)800 049 6602
 enquiries@unigloves.co.uk
 uk.unigloves.com

UNIGLOVES GERMANY
 Arzt- & Klinikbedarf
 Handelsgesellschaft mbH

Camp-Spich-Straße 71
 D-53842 Troisdorf,
 Germany
 +49 (0) 2241-9323-0
 unigloves@unigloves.de
 de.unigloves.com

