

Unigloves Wet Wipes Plus

Surface disinfection

As a medical device: Ready-to-use rapid disinfectant for the prophylactic surface disinfection of non-invasive medical devices Suitable for the medical sector (dental and medical practices, hospitals, care homes, nursing and emergency services) and the food, industrial and public sector.

Composition:

Per 100g:
20.0g 1-propanol; 5.0g 2-propanol

Regulation (EC) No 648/2004:
Contains fragrances

Product Code:

97-2022 Wet Wipes Plus
(80 Sheets)



Benefits

- Excellent material compatibility
- Suitable for disinfecting sensitive surfaces on electronic devices, e.g. screens, touchscreens on tablets and mobile phones, and acrylic glass (PMMA)
- Very broad and rapid microbiological efficacy with a significantly reduced alcohol content – lint-free non-woven fabric
- Phenol-free
- Rapid action
- QAV-free
- Optimal tear-off properties
- The very low alcohol content of just 25% ensures excellent material compatibility, even with materials that are otherwise sensitive to alcohol, such as acrylic glass (PMMA) or imitation leather.
- Virtually residue-free
- Fragrance-free (when used without fragrance oil)
- Short contact time
- VAH-certified
- IHO-listed
- Aldehyde-free

Efficacy and test methods		Contamination	1 min.	5 min.
Bacteria and Candida albicans	EN 13727, EN 13624	High	☞	☞
Tuberculosis pathogens (M. terrae) and mycobacteria (M. avium)	EN 14348	High	☞	☞
'Limited virucidal activity' (enveloped viruses such as HBV, HIV, HCV)	EN 14476	High	☞	☞
'Limited virucidal Plus' (enveloped viruses such as HBV, HIV, HCV and noro-, adeno- and rotaviruses)	EN 14476	Low	☞	☞
Polyoma-SV 40 viruses	RKI/DVV	With contamination	☞	☞
Impregnated wipes (100% PET)				
Bacteria and Candida albicans	EN 16615	High	☞	☞

Other notes

- Only place the packaging in the recycling bin once it has been completely emptied. Dispose of any remaining contents under EAK 07 06 99.
- This product is intended for professional use. Please refer to the safety data sheet or the product label for further information.
- Use disinfectants with care. Always read the label and product information before use.
- The user/patient must report all serious incidents relating to the product to the manufacturer and to the competent authority of the Member State in which the user/patient is based.
- All information is based on our current knowledge and experience and is provided for information purposes only. No warranty and/or guarantee is given in respect of the information provided herein. Information regarding the suitability and use of the products does not constitute a binding agreement regarding the contractual quality or any specific suitability for use of the product. Contractual provisions, in particular agreed product specifications, shall always take precedence. We reserve the right to amend or update the information on this product without prior notice.

Application Instructions

Disinfection of non-invasive medical devices:
Suitable for prophylactic wipe disinfection of medical devices such as dental treatment units, hospital beds, blood pressure monitors, venous catheters, etc. To disinfect, wipe the surface of the medical device with the impregnated wipe. Open the top of the pouch and place it in the dispenser. Pull out the first wipe from the non-woven roll and thread it through the star-shaped opening in the lid. Remove the remaining wipes. After removing the wipes, press the lid firmly onto the dispenser and seal it tightly. Once all the wipes have been used, the dispenser must be cleaned and disinfected in accordance with the reprocessing instructions. Reprocessing should be carried out using a suitable, preferably alcohol-based, disinfectant wipe. All surfaces to be disinfected must be completely saturated with the disinfectant solution. Allow the product to act for the specified contact times. Intended for professional use only. Once the pouch has been opened for the first time, the wipes are usable for a maximum of 28 days.



UNIGLOVES UK
Registered in England No: 4010200
VAT registration 791817787

3 Ambley Green,
Gillingham Business Park,
Gillingham, ME8 0NJ, UK
+44 (0)800 049 6602
enquiries@unigloves.co.uk
uk.unigloves.com

UNIGLOVES GERMANY
Arzt- & Klinikbedarf
Handelsgesellschaft mbH

Camp-Spich-Straße 71
D-53842 Troisdorf,
Germany
+49 (0) 2241-9323-0
unigloves@unigloves.de
de.unigloves.com

