


Proclinic Expert
www.proclinic-products.com

Description

Proclinic Expert VPS Impression Material is an addition silicone impression material that includes a variety of viscosities and two setting times. It features particles with nanometric size and structure in all viscosities to obtain a silicone-nanoparticle combination. The presence of the nanostructure is reported to increase the fluidity of the material, particularly when pressure is applied. All viscosities exhibit a snap set that helps reduce inaccuracies that may occur from movement during setting of the impression. The light, super light, and regular body are available in 50 mL automix cartridges in a normal and/or fast set. Tray material is also available for use with the Pentamix system. Putty is available in a normal and fast set in 250 mL. The working time for normal set is 2 minutes and setting time is 5 minutes. The working time for fast set is 1:30 minutes and setting time is 4 minutes. Twenty-three consultants evaluated **Proclinic Expert VPS Impression Material** in 373 clinical applications. This impression material received a 91% clinical rating.

Product Features

Dispensing all viscosities of **Proclinic Expert VPS Impression Material** from the automix cartridges is consistent and does not require excessive pressure. The light-bodied consistency was rated very good and exhibited good flow around the prepared teeth without slumping. Both the normal set and the fast set have sufficient working times, while the setting times vary from 4 minutes for the fast set to 5:30 for the normal set. When an impression of multiple preparations is needed, the normal set is a good option. Any combination of the viscosities is easily removed from the mouth and all show good tear resistance even in thin sections. Visualizing margin detail is simplified by the excellent color contrast of the different viscosities. The fit of the resultant restorations was judged to be very good. Eighty-seven percent of the consultants rated **Proclinic Expert VPS Impression Material** equivalent to or better than the material they were currently using and 78% would recommend it.

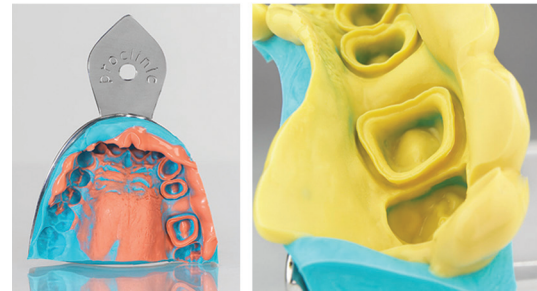

Consultants' Comments

- "I really like the color contrast – margins are easy to read."
- "The light-bodied consistency stays in place and registers margins well."
- "The light-bodied consistency has excellent detail and appears to have good tear strength."
- "Assistants found the putty easier to mix than other brands."
- "Impressions separate easily from their models."

Clinical Tips

- The putty requires mixing with washed hands or using non-latex gloves.
- If the provisional restoration is fabricated before making the impression, completely remove residue from the teeth. Acrylic residue can inhibit the setting of the material.