

# sharpSAFE®

## The all new 5L & 7L Sharps Containers

With DUAL PROTECT **PRO**™

IFU



### To assemble your container

Face the fill line and label forward.



### It just takes four clicks

Four audible clicks on each corner of the lid confirm that the lid and base are assembled correctly and securely.



### Fill in the label before use

This ensures a permanent audit trail.



### Never fill above the line

Always replace your sharps container when full to avoid the risk of needlestick injuries.




### Temporary Closure

To engage, gently press the lid down until the aperture is covered. To release, pull back on the tab and lift the lid until it engages in an open position.



### Final Closure

Once full, permanently close the container. With the container in temporary closure, press the lid down firmly until an audible click is heard.

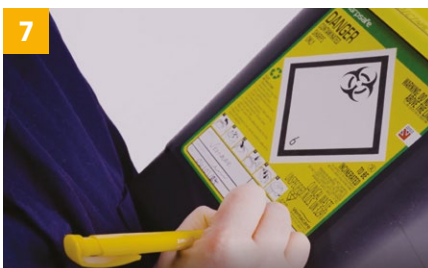

### Final Disposal

Fill in the disposal label and follow local disposal guidance for sharps containers.

Pocket Small Medium Large Specialist Accessories

**Safer today for a sustainable tomorrow.**

1 Western Avenue, Matrix Park, Buckshaw Village, Chorley, PR7 7NB, UK  
+44 (0)1204 529494 hello@vernacare.com vernacare.com

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. © Copyright Vernacare International Limited 2024. All rights reserved. CWM015 REV01



**Vernacare**  
clinical waste  
management