



Scan me for
video

IFU

Plastic
Cardboard



Fill in the label before use.
This ensures a permanent audit trail.




Utilise lid for additional stability
The lid can be utilised as a stand. Place the lid on a flat surface and insert the container into it.



Never overfill the container
Always replace your container when it reaches capacity.



Temporary Closure
To engage, simply place the lid onto the base of the container. Do not apply any pressure.

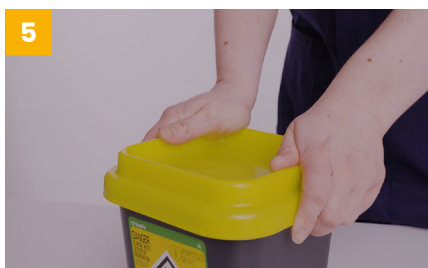

Final Closure
Place both hands on the container lid and apply pressure until an audible click can be heard. Repeat on each corner.



Final Disposal
Fill in the disposal label and follow local disposal guidance for clinical waste containers.

Safer today for a sustainable tomorrow.

1 Western Avenue, Matrix Park, Buckshaw Village, Chorley, PR7 7NB, UK
+44 (0)1204 529494 hello@vernacare.com vernacare.com

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. © Copyright Vernacare International Limited 2024. All rights reserved. CWM077 REV00



Vernacare
clinical waste
management