



Scan me for
video

IFU

Plastic
Cardboard



Fill in the label before use

This ensures a permanent audit trail.



Never overfill the container

Always replace your container when it reaches capacity.




Temporary Closure

To engage, simply place the lid onto the base of the container. Do not apply any pressure.



Final Closure

Place both hands on the container lid and apply pressure until an audible click can be heard. Repeat on each corner.

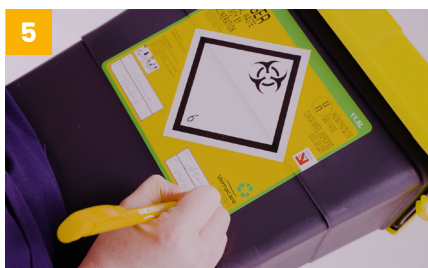

Final Disposal

Fill in the disposal label and follow local disposal guidance for clinical waste containers.

Safer today for a sustainable tomorrow.

1 Western Avenue, Matrix Park, Buckshaw Village, Chorley, PR7 7NB, UK
+44 (0)1204 529494 hello@vernacare.com vernacare.com

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. © Copyright Vernacare International Limited 2024. All rights reserved. CWM078 REV00



Vernacare
clinical waste
management