



Scan me for
video


IFU

Plastic
Cardboard



Close and seal the bag insert

Remove the string from inside the container. Twist the bag so it is securely sealed with the string.

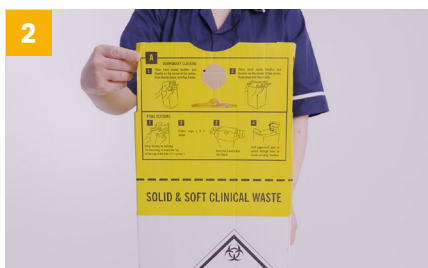

Locate the indicators

Each side has alphabetical indicators: A, B, and C for closing the container.



Fold flap A of the container

Place your hand inside the handles and your thumb on the corner. Push thumbs down until flap A folds.



Fold flap B of the container

Place your hand inside the handles and your thumb on the corner. Push thumbs down until flap B folds.



Insert tab C into the container and fill in label

Bend flap C and place it into flap A. Remember to fill in the label located on flap B at this point.



Secure the handles

Fold the outer part of the carton through the holes to create carry handles. Repeat on both sides.



Final Disposal

Fill in the disposal label. Follow local disposal guidance for clinical waste containers.

Safer today for a sustainable tomorrow.

1 Western Avenue, Matrix Park, Buckshaw Village, Chorley, PR7 7NB, UK
+44 (0)1204 529494 hello@vernacare.com vernacare.com

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. © Copyright Vernacare International Limited 2024. All rights reserved. CWM083 REV00



Vernacare
clinical waste
management