



Scan me for video

IFU

Plastic
Cardboard



To open the cardboard container
Hold two corners of the flat packed box and pull them apart in opposite directions. Place your arm inside the container to help separate if needed.




Lock the container open
Turn the carton over and push the sides until the automatic bottom locks open.



Fill in the label before use
This ensures a permanent audit trail.



Place the container into a plinth
All cardboard containers should be placed into a plinth to prevent damage to the base.

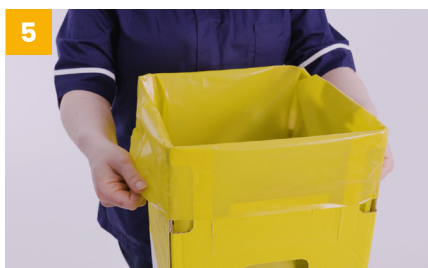

Secure the bag insert
Open the bag and place around the perimeter of the carton. Secure in place by folding over each edge of the container.



Do not overfill
Do not fill above the fill line or weight limitation.

Safer today for a sustainable tomorrow.

1 Western Avenue, Matrix Park, Buckshaw Village, Chorley, PR7 7NB, UK
+44 (0)1204 529494 hello@vernacare.com vernacare.com

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. © Copyright Vernacare International Limited 2024. All rights reserved. CWM081 REV00



Vernacare
clinical waste
management