



GENTLE.SUPPORTIVE.TRUSTED

Comfi™ Tubular Bandages and Garments for healthcare professionals



Vernacare
surgical solutions

Contents

Why Choose Comfi®	3
ComfiFAST™ Tubular Bandages	4
ComfiFAST™ MultiStretch™ Tubular Bandages	6
ComfiWEAR™	8
ComfiKIDS™	10
Dry Wrapping Guide	12
Comfi™ Single-Use Tubular Bandages	14
Comfi™ Range Technical Information	16


Vernacare has been working with the NHS for over 50 years and we’ve been the brand leader in tubular bandages since 2013. **Our Comfi® Range offers a selection of high-quality tubular bandages and garments**, developed by a team of woundcare experts.

We’re here to support you – offering you the lowest costs, our products have been developed with benefits to support you and your patients. ComfiFAST™ tubular bandages and garments are made in the UK using high quality materials and SteamSoft™ Fabric, resulting in highly robust but soft and comfortable fabric that is ready to wear with no pins or tapes required...

The result – quick and easy application for both the healthcare professional and the patient.

WHY CHOOSE COMFI®

As the leading brand within the NHS and wider care sector, the Comfi® Range supports individuals and healthcare workers in delivering excellent standards of care, offering an extensive range for all your needs.



EST
2002

Established brand for over 21 years



MADE IN
BRITAIN

Dedicated team of woundcare experts



MADE IN
BRITAIN

Manufactured in the UK




Using high quality materials



SteamSoft™ Fabric for extra comfort



MultiStretch™ Technology – extra freedom of movement



Full range availability



Cost effective offer – best price in market



46% of UK prescribers choose Comfi®






ComfiFAST™ Tubular Bandages

ComfiFAST™ tubular bandages are commonly used for dressing retention, dry wrapping of atopic eczema and skin covering for fragile skin.

When used for dressing retention, ComfiFAST™ holds the primary wound dressing in place, providing an extra layer of protection during the healing process.

These colour coded bandages have been produced using SteamSoft™ Technology and light, breathable fabrics, for patient comfort whilst offering single stretch in the radial direction.


INDICATED FOR :

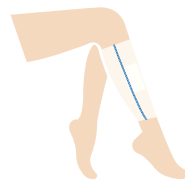

Dressing retention



Eczema, dry skin & psoriasis

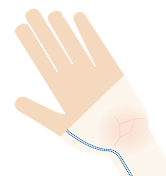

Covering fragile skin

PRODUCT OVERVIEW:



Wash and reuse



Holds primary dressing



Light & breathable fabric



Variety of widths and lengths



No need for pins or tape



Extra layer of protection

MATERIAL:

Viscose, Elastane, Polyamide



EXTRA LAYER OF PROTECTION



STEAMSOFT TECHNOLOGY :

Our SteamSoft™ Technology is used on all our products, leaving the fabrics softer and wearable straight from the box.




For more information and to order samples visit [vernacare.com](https://www.vernacare.com)



ComfiFAST™ MultiStretch™ Tubular Bandages

ComfiFAST™ MultiStretch™ tubular bandages are produced using the same light and breathable fabrics as ComfiFAST™ with added MultiStretch™ Technology.

INDICATED FOR :

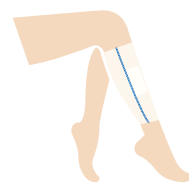

Dressing retention




Primary & secondary lymphoedema




Eczema, dry skin & psoriasis



Under compression bandaging

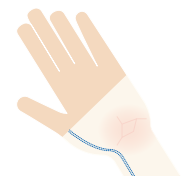

Covering fragile skin

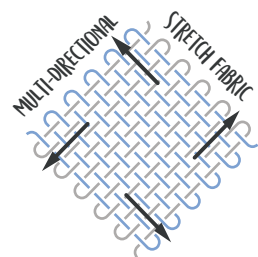

MULTI-STRETCH TECHNOLOGY

ComfiFAST™ MultiStretch™ bandages stretch in the radial, longitudinal and diagonal directions for increased freedom of movement and improved patient comfort.

IDEAL FOR JOINTS



PRODUCT OVERVIEW:



Wash and reuse



Variety of widths and lengths



Holds primary dressing



No need for pins or tape



Light & breathable fabric



Extra layer of protection

MATERIAL: Viscose, Elastane, Polyamide

For more information and to order samples visit [vernacare.com](https://www.vernacare.com)



ComfiWEAR™ Ready to Wear Garments

ComfiWEAR™ ready to wear adult garments provide the ideal solution for covering the skin when using steroids, heavy emollients or ointments for the treatment of common conditions such as eczema, dermatitis and psoriasis.

ComfiWEAR™ adult garments have been developed to be used in conjunction with oils and emollients prescribed by healthcare professionals. By incorporating MultiStretch™ Technology, the garments are constructed to stretch in multiple directions, meaning they are easy to apply, offer additional comfort and are discreet to wear under clothing.

With superior SteamSoft™ Technology, our garments are not only ready to wear but also suitable for machine washing, allowing them to be re-used again and again whilst still retaining their softness.

ComfiWEAR™ adult garments are also an ideal solution for securing dressings in difficult areas of the body, such as the chest and groin.


INDICATED FOR :



Dressing retention



Eczema, dry skin & psoriasis

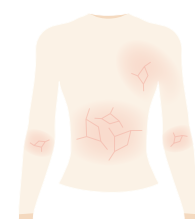

Covering fragile skin



PRODUCT OVERVIEW:



Simple, safe and easy-to-use



Seams on the outside



Creates a barrier to scratching



Unobtrusive when worn under clothing



Prevents transfer of creams & emollients to clothing and bedding



Helps support patient concordance



For more information and to order samples visit [vernacare.com](https://www.vernacare.com)



ComfiKIDS™ Ready to Wear Garments

ComfiKIDS™ ready to wear garments provide the ideal solution for covering the skin when using steroids, heavy emollients or ointments for the treatment of common conditions such as eczema, dermatitis and psoriasis.

ComfiKIDS™ garments have been developed to be used in conjunction with oils and emollients prescribed by healthcare professionals. By incorporating MultiStretch™ Technology, the garments are constructed to stretch in multiple directions, meaning they are easy to apply by the professional or parent. The garments offer additional comfort and are discreet to wear under clothing.

With superior SteamSoft™ Technology, our garments are not only ready to wear but also suitable for machine washing, allowing them to be re-used again and again whilst still retaining their softness.

ComfiKIDS™ garments are also an ideal solution for securing dressings in difficult areas of the body, such as the chest and groin.

The garments are designed to fit each area of the body, ranging from a clava for the head through to socks for the feet, allowing the patient to live an active day and a restful night!



INDICATED FOR :



Dressing retention



Eczema, dry skin & psoriasis



Covering fragile skin

A COMFORTABLE NIGHT

PRODUCT OVERVIEW:



Simple, safe and easy-to-use



Seams on the outside



Creates a barrier to scratching



Unobtrusive when worn under clothing



Prevents transfer of creams & emollients to clothing and bedding



Helps support patient concordance

MATERIAL: Viscose, Elastane, Polyamide

Technical information, size availability and product codes, can be found on pages 14 & 15.

MEET THE COMFIKIDS

All of these children have skin conditions and wear our garments to help them lead healthy, happy lives.

JOSEF

“I’m 10 years old, I play football twice a week and want to be a footballer when I’m older!”



GRACIE

Gracie is nearly one year old. Gracie is always on the move, she has recently taken her first steps and is so proud of herself. Her parents are also proud of her, she now sleeps through the night, so a peaceful night for mummy and daddy, as well as Gracie!



RAJ

Raj is nearly 3 years old, he attends nursery and loves to join in playing games with his friends but he also loves cuddles before bedtime!



ERIN

“Hi, I’m Erin. I’m preparing for my GCSEs, but when I’m not at school or revising, I like to hang out with my friends, we love to have sleepovers!”



TIANA

“My favourite thing to do is dancing. I’m 8 years old and I started dance lessons when I was 4. It’s really tiring but lots of fun!”






DRY WRAPPING GUIDE

Dry wrap therapy is a technique developed to treat the symptoms of Atopic Eczema. Using this technique with ComfiWEAR™ or ComfiKIDS™ garments, will reduce itching, improve sleep and increase comfort – helping patients to carry on with a normal life, night and day.



Step 1 – Choose the garments to be used for each area of the body that needs to be wrapped. If it's the legs, you will need one pair of leggings or tights (dependent on age).

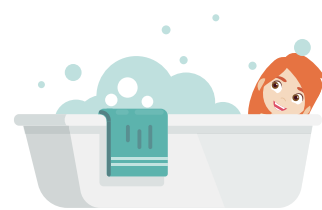

Step 2 – Bathe with the appropriate bath oil or emollient recommended by your healthcare professional. After bathing, pat the skin dry and apply any steroids or emollients that have been prescribed or recommended.




Step 3 – Put on a single dry layer of the garment over the emollients and dress as normal. (Remember the label and seams should be on the outside).

For more information and to order samples visit [vernacare.com](https://www.vernacare.com)



INDICATED FOR :

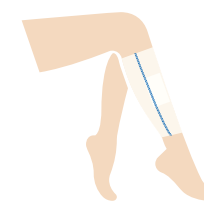

Dressing retention

ComfiNETTE™, ComfiTUBE™ and ComfiGAUZ™ Tubular Bandages

ComfiNETTE™, ComfiTUBE™ and ComfiGAUZ™ are a range of single-use cost-effective tubular bandages for dressing retention. The range is available in a variety of widths and lengths to fit all areas of the body, ranging from fingers and toes to an adult torso.

MATERIAL:

ComfiNETTE™ is made from **100% viscose**, ideal for frequent dressing changes.

ComfiTUBE™ is made from **100% unbleached cotton**, containing natural oils making the bandage water repellent.

ComfiGAUZ™ is made from **100% bleached cotton**, ideal for use where natural fibres are preferred.

PRODUCT OVERVIEW:



Simple, safe and easy-to-use



Light & breathable fabric



Variety of widths and lengths






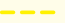

Cost effective

Technical information, size availability and product codes, can be found on pages 14 & 15.








TECHNICAL INFORMATION



WIDTH (unstretched)	Application	Limb circumference	Ref 1m	Ref 3m	Ref 5m	Ref 10m
3.5cm Red Line 	Small Limbs	8-15cm	F12 EGP063	-	-	F15 EGP005
5.0cm Green Line 	Medium Limbs	10-25cm	F22	F23	F24	F25 EGP006
7.5cm Blue Line 	Large Limbs	20-45cm	F32 EGP067	F33	F34 EGP069	F35 EGP007
10.75cm Yellow Line 	Extra Large Limbs, Head, Trunk (Child)	35-65cm	F42 EGP070	F43 EGP071	F44 EGP072	F45 EGP008
17.5cm Beige Line 	Trunk (Adult)	50-120cm	F52	-	-	F55 EGP009



WIDTH (unstretched)	Application	Limb circumference	Ref 1m	Ref 3m	Ref 5m	Ref 10m
3.5cm Red Line 	Small Limbs	8-15cm	FMS12	-	-	FMS15 EGP036
5.0cm Green Line 	Medium Limbs	10-25cm	FMS22	FMS23	FMS24 EGP172	FMS25 EGP037
7.5cm Blue Line 	Large Limbs	20-45cm	FMS32	FMS33	FMS34	FMS35 EGP038
10.75cm Yellow Line 	Extra Large Limbs, Head, Trunk (Child)	35-65cm	FMS42	FMS43	FMS44	FMS45 EGP039
17.5cm Beige Line 	Trunk (Adult)	50-120cm	FMS52	-	-	FMS55



Product	Size : Limb Circumference	REF	NPC
Child Vest	6-24 months	CV24	EGC014
	2-5 years	CV5	EGC015
	5-8 year	CV8	EGC016
	8-11 years	CV11	EGC017
	11-14 years	CV14	EGC018
Child Tights	6-24 months	CT24	EGC022
Child Leggings	2-5 years	CL5	EGC023
	5-8 years	CL8	EGC024
	8-11 years	CL11	EGC025
	11-14 years	CL14	EGC026
Child Clava	6 months-5 years	CC5	EGC012
	5-14 years	CC14	-
Child Mittens	Up to 24 months (with thumb hole)	CM24	EGC019
	8 years (with thumb hole)	CM8	EGC020
	8-14 years (without thumb hole)	CM14	EGC021
Child Socks	up to 8 years	CS8	EGC027
	8-14 years	CS14	EGC028



Product	Size	Limb Circumference	REF	NPC
Adult Vest	Small	Chest : 71-91cm (28"-36")	CV20	-
	Medium	Chest : 91-117cm (36"-46")	CV21	EGC037
	Large	Chest : 117-152cm (46"-60")	CV22	EGC038
Adult Leggings	Small	Waist / Hips : 71-91cm (28" - 36")	CL20	-
	Medium	Waist / Hips : 91-117cm (36"-46")	CL21	EGC034
	Large	Waist / Hips : 117-152cm (46"-60")	CL22	EGC035



Size	Application	ComfiNETTE™ 20m	ComfiTUBE™ 20m	ComfiGAUZ™ 20m
00	Toes	N006 EGJ045	-	Z006 EGJ051
01	Fingers and toes	N016 EGJ041	T016 EGJ055	Z016 EGJ035
12	Bulky dressing for fingers and toes	N126 EGJ042	T126 EGJ056	Z126 EGJ036
34	Children's limbs / adult hands	-	-	Z346 EGJ052
56	Adult limbs	N566 EGJ043	T566 EGJ058	Z566 EGJ037
78	Large adult limbs	N786 EGJ044	T786 EGJ039	Z786 EGJ038
T1	Small trunk / adult head	NT16 EGJ047	TT16	ZT16 EGJ053
T2	Large trunk	-	TT26 EGJ060	-





Comfi™



Vernacare Limited, 1 Western Avenue, Matrix Park
Chorley, Lancashire, PR7 7NB, United Kingdom.
Telephone: +44 (0)1204 529494 Fax: +44 (0)1772 299901
Email: hello@vernacare.com [vernacare.com](https://www.vernacare.com)



Comfi®, ComfiFAST™, MultiStretch™ & SteamSOFT™, ComfiWEAR™, ComfiKIDS™, ComfiNETTE™, ComfiTUBE™ and ComfiGAUZ™ are trademarks of Vernacare Limited © Vernacare Limited 2024.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice.

WC530 REV02

Vernacare
surgical solutions