

GENTLE. SUPPORTIVE. TRUSTED



ComfiKIDS™

Ready to Wear Garments



**Vernacare**  
surgical solutions



**NO 1  
MARKET LEADER  
IN THE UK**

## ComfiKIDS™ Ready to Wear Garments

ComfiKIDS™ ready to wear garments provide the ideal solution for covering the skin when using steroids, heavy emollients or ointments for the treatment of common conditions such as eczema, dermatitis and psoriasis.

ComfiKIDS™ garments have been developed to be used in conjunction with oils and emollients prescribed by Healthcare Professionals. By incorporating MultiStretch Technology™, the garments are constructed to stretch in multiple directions, meaning they are easy to apply by the professional or parent. The garments offer additional comfort and are discreet to wear under clothing.

With superior SteamSoft Technology™, our garments are not only ready to wear but also suitable for machine washing, allowing them to be re-used again and again whilst still retaining their softness.

ComfiKIDS™ garments are also an ideal solution for securing dressings in difficult areas of the body, such as the chest and groin.

The garments are designed to fit each area of the body, ranging from a clava for the head through to socks for the feet, allowing the patient to live an active day and a restful night!



## PRODUCT OVERVIEW:



Simple, safe and easy-to-use



Seams on the outside



Creates a barrier to scratching



Unobtrusive when worn under clothing



Prevents transfer of creams & emollients to clothing and bedding



Helps support patient concordance

**MATERIAL:** Viscose, Elastane, Polyamide

**Technical information, size availability and product codes, can be found on page 7**

## INDICATED FOR :



Dressing retention



Eczema, dry skin & psoriasis



Covering fragile skin






## DRY WRAPPING GUIDE

Dry wrap therapy is a technique developed to treat the symptoms of Atopic Eczema. Using this technique with ComfiWEAR™ or ComfiKIDS™ garments, will reduce itching, improve sleep and increase comfort – helping patients to carry on with a normal life, night and day.



**Step 1** – Choose the garments to be used for each area of the body that needs to be wrapped. If it's the legs, you will need one pair of leggings or tights (dependent on age).

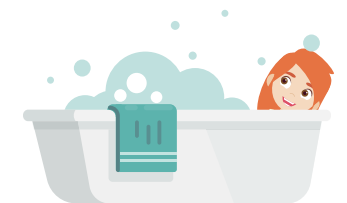

**Step 2** – Bathe with the appropriate bath oil or emollient recommended by your healthcare professional. After bathing, pat the skin dry and apply any steroids or emollients that have been prescribed or recommended.



**Step 3** – Put on a single dry layer of the garment over the emollients and dress as normal. (Remember the label and seams should be on the outside).

For more information and to order samples visit **vernacare.com**



# MEET THE COMFIKIDS

All of these children have skin conditions and wear our garments to help them lead healthy, happy lives.

### JOSEF

“I’m 10 years old, I play football twice a week and want to be a footballer when I’m older!”

### ERIN

“Hi, I’m Erin. I’m preparing for my GCSEs, but when I’m not at school or revising, I like to hang out with my friends, we love to have sleepovers!”

### RAJ

Raj is nearly 3 years old, he attends nursery and loves to join in playing games with his friends but he also loves cuddles before bedtime!

### GRACIE


Gracie is nearly one year old. Gracie is always on the move, she has recently taken her first steps and is so proud of herself. Her parents are also proud of her, she now sleeps through the night, so a peaceful night for mummy and daddy, as well as Gracie!

### TIANA

“My favourite thing to do is dancing. I’m 8 years old and I started dance lessons when I was 4. It’s really tiring but lots of fun!”

# LOWEST COST ON DRUG TARIFF



Product	Size : Limb Circumference	REF	NPC
 Child Vest	6 - 24 months	CV24	EGC014
	2 - 5 years	CV5	EGC015
	5 - 8 year	CV8	EGC016
	8 - 11 years	CV11	EGC017
	11 - 14 years	CV14	EGC018
 Child Tights	6 - 24 months	CT24	EGC022
 Child Leggings	2 - 5 years	CL5	EGC023
	5 - 8 years	CL8	EGC024
	8 - 11 years	CL11	EGC025
	11 - 14 years	CL14	EGC026
 Child Clava	6 months - 5 years	CC5	EGC012
	5 - 14 years	CC14	-
 Child Mittens	Up to 24 months (with thumb hole)	CM24	EGC019
	8 years (with thumb hole)	CM8	EGC020
	8-14 years (without thumb hole)	CM14	EGC021
 Child Socks	up to 8 years	CS8	EGC027
	8-14 years	CS14	EGC028



Vernacare Limited, 1 Western Avenue, Matrix Park  
Chorley, Lancashire, PR7 7NB, United Kingdom.  
Telephone: +44 (0)1204 529494 Fax: +44 (0)1772 299901  
Email: [hello@vernacare.com](mailto:hello@vernacare.com) **vernacare.com**



Comfi®, ComfiFAST™, MultiStretch™ & SteamSOFT™, ComfiWEAR™, ComfiKIDS™, ComfiNETTE™, ComfiTUBE™ and ComfiGAUZ™ are trademarks of Vernacare Limited © Vernacare Limited 2024.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice.

WC651 REV01

**Vernacare**  
surgical solutions