


Instructions For Use

Bed Bathing Washcloths and Washmitts


The Conti® Waterless Bathing range provides a convenient and effective solution for bed bathing. Suitable for everyday use, the large heavyweight cloths and mitts cleanse and condition, whilst increasing skin moisturisation. Intended to be used individually, with one single-use cloth per anatomical region, to help minimise the risk of cross-contamination.

1

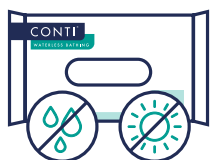

Keep product dry and out of direct sunlight.

2

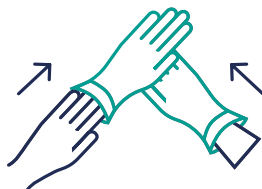

Wash/dry hands and apply clean protective apron and gloves.

3



Remove wipes/mitts from the dedicated Conti® Warmer, or use at room temperature.

4



Note how many wipes/mitts are contained in the pack and check temperature.

WARNINGS: Do not use if too warm; Check on-pack instructions; Once pack is opened, do not re-warm and dispose of any unused product.


5



Remove and use one disposable washcloth or mitt at a time.

Ensure individual's privacy and reassure them.

6

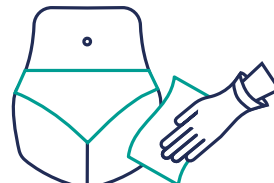

Cleanse the body in the order shown on the diagram, using one wipe per area.

7




The wipes/mitts will cleanse and condition the skin.

8



Allow skin to dry naturally, do not rinse.

9

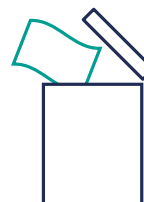

Check the pack for disposal guidance, and discard used wipes/mitts and PPE in line with local policies.

| Product Code | Description | Perfume | Wipe Size | Pack Qty | Case Qty |
|--------------|-----------------------|--------------------|-----------|----------|----------|
| BB05UPN | Bed Bathing Washcloth | Fragrance Free | 30 x 22cm | 5 | 68 |
| BB08N | Bed Bathing Washcloth | Lightly Fragranced | 30 x 22cm | 8 | 42 |
| BB08UPN | Bed Bathing Washcloth | Fragrance Free | 30 x 22cm | 8 | 42 |
| BB10N | Bed Bathing Washcloth | Lightly Fragranced | 30 x 22cm | 10 | 34 |
| BB10UPN | Bed Bathing Washcloth | Fragrance Free | 30 x 22cm | 10 | 34 |
| WM08N | Bed Bathing Washmitts | Lightly Fragranced | 22 x 15cm | 8 | 30 |
| WM08UPN | Bed Bathing Washmitts | Fragrance Free | 22 x 15cm | 8 | 30 |



DISCOVER MORE:
[vernacare.com/conti-wet-wipes](https://www.vernacare.com/conti-wet-wipes)

ORDER SAMPLES:
samples@vernacare.com



Vernacare Limited, 1 Western Avenue, Matrix Park, Chorley, Lancashire, PR7 7NB, United Kingdom.
T: +44 (0)1772 299900 F: +44 (0)1772 299901 Email: info@vernacare.com www.vernacare.com

Vernacare

