




Scan me for
video

IFU

Pocket Small Medium Large Specialist Accessories

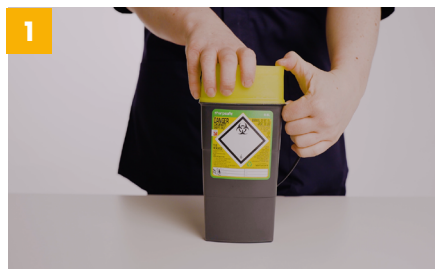

To assemble your container

The pocket range arrive pre-assembled. We recommend pulling out the safety handle before use.



Fill in the label before use

This ensures a permanent audit trail.



Never fill above the line

Always replace your sharps container when full to avoid the risk of needlestick injuries.



Temporary Closure

To engage, gently press the lid down until the aperture is covered. This feature should always be engaged when not in use.



Final Closure

Once full, the container should be permanently closed. With the container in temporary closure, press the lid down firmly until an audible click is heard.



Final Disposal

Fill in the disposal label and follow local disposal guidance for sharps containers.

Safer today for a sustainable tomorrow.

1 Western Avenue, Matrix Park, Buckshaw Village, Chorley, PR7 7NB, UK
+44 (0)1204 529494 hello@vernacare.com vernacare.com

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. © Copyright Vernacare International Limited 2024. All rights reserved. CWM050 REV00



Vernacare
clinical waste
management