

IFU

Pocket Small Medium Large Specialist Accessories



To assemble your container
Face the fill line and label forward.



Attach the lid with just four clicks
Place the lid on top of the container and press firmly on each corner of the lid until you hear an audible click.



Fill in the label before use
This ensures a permanent audit trail.




Never fill above the line
Always replace your sharps container when full to avoid the risk of needlestick injuries.



Temporary Closure
To engage, close the lid until it covers the aperture and a click is heard. To release, press down on the tab, lift lid until it engages in an open position.



Final Closure
Once full, the container should be permanently closed. With the container in temporary closure, press the lid down firmly until an audible click is heard.

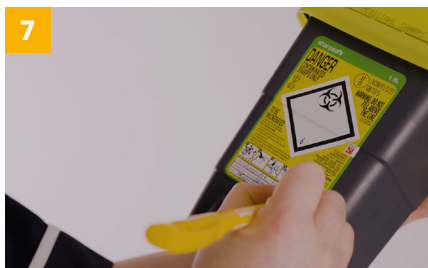

Final Disposal
Fill in the disposal label and follow local disposal guidance for sharps containers.

Safer today for a sustainable tomorrow.

1 Western Avenue, Matrix Park, Buckshaw Village, Chorley, PR7 7NB, UK
+44 (0)1204 529494 hello@vernacare.com vernacare.com

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. © Copyright Vernacare International Limited 2024. All rights reserved. CWM054 REV00



Vernacare
clinical waste
management