



Scan me for
video

IFU

Pocket Small Medium Large Specialist Accessories



To assemble your container

Face the fill line and label forward.



Attach the lid with just four clicks

Place the lid on top of the container and press firmly on each corner of the lid until you hear an audible click.



Fill in the label before use

This ensures a permanent audit trail.



Never fill above the line

Always replace your sharps container when full to avoid the risk of needlestick injuries.



Protected Access Feature

Insert the sharp as normal, then push down on the protected access shield to dispose of the sharp. Temporary closure should always been engaged when container is not in use.



Final Closure

Once full, the container should be permanently closed. With the container in temporary closure, press down firmly on the front of the flap until an audible click is heard.



Final Disposal

Fill in the disposal label and follow local disposal guidance for sharps containers.

Safer today for a sustainable tomorrow.

1 Western Avenue, Matrix Park, Buckshaw Village, Chorley, PR7 7NB, UK
+44 (0)1204 529494 hello@vernacare.com vernacare.com

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. © Copyright Vernacare International Limited 2024. All rights reserved. CWM062 REV00



Vernacare
clinical waste
management