



Scan me for
video

IFU

Pocket Small Medium Large Specialist Accessories



To assemble your container
Container arrives pre-assembled.



Insert container into an XL Trolley
This container should be used with a trolley to guarantee maximum stability.



Insert protected access feature
If you are using this feature, insert the shield into the base of the container until an audible click can be heard.



Fill in the label before use
This ensures a permanent audit trail.



Never fill above the line
Always replace your sharps container when full to avoid the risk of needlestick injuries.



Closure
When the container is not in use, close the lid. Once full, close the lid and press down firmly on all four corners until an audible click is heard.



Final Disposal
Fill in the disposal label and follow local disposal guidance for sharps containers.

Safer today for a sustainable tomorrow.

1 Western Avenue, Matrix Park, Buckshaw Village, Chorley, PR7 7NB, UK
+44 (0)1204 529494 hello@vernacare.com vernacare.com

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. © Copyright Vernacare International Limited 2024. All rights reserved. CWM065 REV00



Vernacare
clinical waste
management