



IFU



1 Assemble your Sharpsafe® container
Assemble your container in line with the relevant IFU. Ensure you have selected the right size container for the NPSD tray.



2 Ensure the label is facing forward
This ensures the fill line and safety information is always visible.



3 Insert container into the tray
With the container in temporary closure, place the container into the smaller section of the tray.



4 Gather your equipment
Utilise the tray to carry any items you may need.



5 Carry the tray to the patient
Carry the tray, and place on a suitable flat surface near the patient. Carry out the procedure and dispose of sharp at point of use.



6 Clean tray after each use
Clean the tray after each use. Wash thoroughly with hot water and general purpose detergent, or a wipe alternative. Dry thoroughly with paper towels.

Pocket Small Medium Large Specialist Accessories

Safer today for a sustainable tomorrow.

1 Western Avenue, Matrix Park, Buckshaw Village, Chorley, PR7 7NB, UK
+44 (0)1204 529494 hello@vernacare.com vernacare.com

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. © Copyright Vernacare International Limited 2024. All rights reserved. CWM067 REV00



Vernacare
clinical waste
management