



IFU



Assemble your Sharpsafe® container
Assemble your container in line with the relevant IFU.




When inserting a 24L or 30L container
Place the container into the trolley with the label facing forward.



When inserting a 9L or 13L container
Insert the container sideways into the trolley. Place another container in the opposite side.



Release the brake
Remove the brake by clicking it into the unlocked position with your foot.

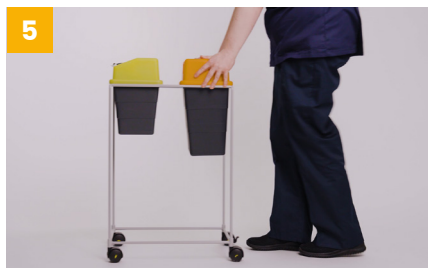

Move the trolley to the patient
At patient, apply the brake. Perform the procedure and dispose of sharp. Place the container in temporary closure before moving.



Return the trolley
Return the trolley to the storage location and apply the brake. If the container contents is full, replace the container.

Pocket Small Medium Large Specialist Accessories

Safer today for a sustainable tomorrow.

1 Western Avenue, Matrix Park, Buckshaw Village, Chorley, PR7 7NB, UK
+44 (0)1204 529494 hello@vernacare.com vernacare.com

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. © Copyright Vernacare International Limited 2024. All rights reserved. CWM073 REV00



Vernacare
clinical waste
management