



Scan me for  
video

IFU



### Assemble your Sharpsafe® container

Assemble your container in line with the relevant IFU.



### When inserting a 24L or 30L container

Place the container into the trolley with the label facing forward.



### When inserting a 9L or 13L container

Insert the container sideways into the trolley. Place another container in the opposite side.



### Release the brake

Remove the brake by clicking it into the unlocked position with your foot.



### Move the trolley to the patient

At patient, apply the brake. Perform the procedure and dispose of sharp. Place the container in temporary closure before moving.



### Return the trolley

Return the trolley to the storage location and apply the brake. If the container contents is full, replace the container.

Pocket Small Medium Large Specialist Accessories

**Safer today for a sustainable tomorrow.**

1 Western Avenue, Matrix Park, Buckshaw Village, Chorley, PR7 7NB, UK  
+44 (0)1204 529494 hello@vernacare.com vernacare.com

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. © Copyright Vernacare International Limited 2024. All rights reserved. CWM074 REV00



**Vernacare**  
clinical waste  
management