



Scan me for
video

IFU

Pocket Small Medium Large Specialist Accessories



Assemble your Sharpsafe® container
Assemble your Sharpsafe® container as per the assembly instructions.



Secure your stand using glue tabs
Remove the covering and apply your glue tabs to the ridges on the bottom of the stand. Remove the second covering.



Secure your stand suction cups
Insert the suction cups into the four holes on the bottom of the stand.



Secure your stand to a flat surface
Place secure your stand a flat surface and apply pressure to secure your stand.




Insert your container
Insert your container into the stand, ensuring the fill line and label are facing forward.



Final closure
Once your container is full, permanently close your sharps container.



Remove your container
Place one hand on the stand and your other hand on the handle of the container. Pull the handle gently until the container is out of the stand. Do not touch the body of the container when removing.

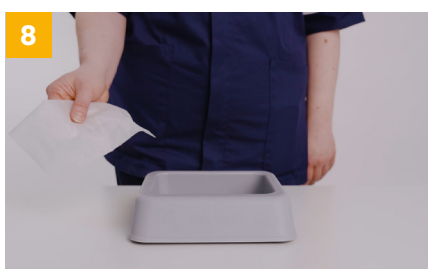

Cleaning your stand
Clean the stand before inserting a new container. We recommend cleaning the stand as per your local cleaning protocols.

Safer today for a sustainable tomorrow.

1 Western Avenue, Matrix Park, Buckshaw Village, Chorley, PR7 7NB, UK
+44 (0)1204 529494 hello@vernacare.com vernacare.com

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. © Copyright Vernacare International Limited 2024. All rights reserved. CWM075 REV00



Vernacare
clinical waste
management